

DENTAL 2023





MEDICAL MATERIALS

DETAX
HIGHEND MEDICAL MATERIALS

TRUST & INTEGRITY
INTERCULTURAL COMPETENCE
ENTHUSIASM FOR POLYMER CHEMISTRY

WERK STOFF 4.0

ADVANCED, INNOVATIVE MATERIALS ARE
THE DRIVING FORCE OF PRODUCT DEVELOPMENT
IN THE FIELD OF MEDICAL DEVICES
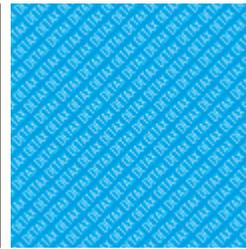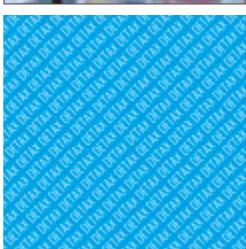
<p>We develop biocompatible silicones and high-performance polymers for medical applications and 3D printing.</p>			<p>Our Business relationships are built upon appreciation, integrity and intercultural expertise.</p>
	<p>Our enthusiasm and innovative strength lies in every one of the products you will get to know over the next pages</p>	<p>State-of-the-art warehouse logistics and lean IT processes support the global flow of goods and consistent availability.</p>	
			<p>The unique physical properties of polymer compounds offer nearly unlimited possibilities for tomorrow's products.</p>
	<p>Modern materials are the crucial driver for the development of novel medical devices.</p>		

DETAX GMBH · ETTLINGEN, GERMANY · MARCH 2023

CONTENTS

6 - 7 COMMUNICATION & SERVICE

9 USER MATRIX A-SILICONES

20 USER MATRIX C-SILICONES

53 USER MATRIX 3D RESINS

11 - 71 PRODUCTS

72 INDEX



IMPRESSION TAKING

8

A-SILICONE
BITE REGISTRATION
C-SILICONE



SOFT RELINING

31


COLD CURING MATERIAL
HEAT CURING MATERIAL
SILICONE GLOSS VARNISH
ACCESSORIES



PREVENTION & REGENERATION

43

SEALING OF ROOT DENTINE
FISSURE SEALING
MOUTH RINSING SOLUTION

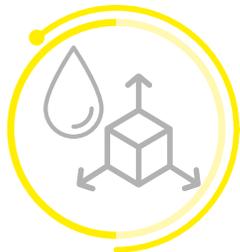

3D RESINS

49

TOTAL DENTURES
CROWNS & BRIDGES
ORTHODONTIC BASE COMPONENTS
BRACKET TRANSFER TEMPLATES
SPLINTS
TRAYS & PLATES
CASTINGS
MODEL PRODUCTION
GINGIVAL MASK

26 ACCESSORIES IMPRESSION TAKING

AUTOMIX
INLAY TRAYS
TRAY ADHESIVE
ACCESSORIES



35 TEMPORARY TREATMENT

C & B COMPOSITE
FIXATION
IMPLANT CEMENT
REPAIR
SEALING



47 LIQUIDS, SOLUTIONS

TOPICAL APPLICATION LIQUID
SPRAY SOLUTION



60 LABORATORY

LIGHT CURING POLYMERS
PUTTY
ISULATION
GINGIVAL MASK
RELINING
RESOURCES




SERVICES

Do you have any questions regarding our products, a shipment or another matter? You can reach our service-team Monday to Friday between 8 a.m. and 5 p.m.

Do you have technical questions about a certain product, an application technique or chemistry? Our technicians are happy to assist you.

At our Website www.detax.com, you will find even more detailed product information, international events, contact persons, news and many more useful information.
Now NEW: Live Chat with experts at www.detax.com

Do you need documents, such as studies, specific information, MSDS, declarations of conformity or instructions for use? All documentations are available for you.

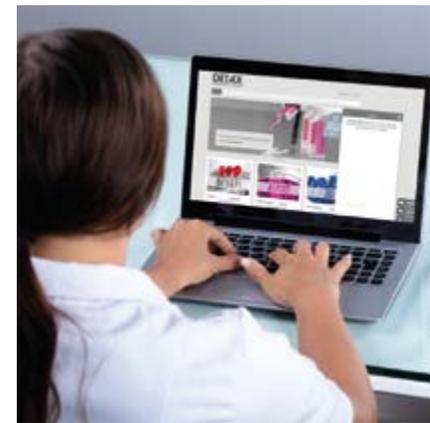

CUSTOMER SERVICE

service@detax.com
Phone +49 7243 - 510-103
Phone +49 7243 - 510-104
Fax +49 7243 - 510-100

PRODUCT APPLICATION

application@detax.com
3Dapplication@detax.com
Phone +49 7243 - 510-146
Fax +49 7243 - 510-100

REGULATORY AFFAIRS

regulatory.affairs@detax.com
Phone +49 7243 - 510-268
Fax +49 7243 - 510-243

MDR

medi.guide@detax.com
Phone +49 7243 - 510-208
Fax +49 7243 - 510-222

EXPERT LIVE CHAT

www.detax.com





CERTIFICATION

We support our customers in all matters pertaining to medical devices. Our specialists have long-standing experience in the interpretation and implementation of international certification standards and individual country requirements. We will give you competent advice on required registrations for the market access of DETAX brands, OEM or custom-made materials. We will supply your quality management with the required documentation (e.g. ISO certificates, declarations of conformity, product labelling, safety data sheets or technical data sheets, etc.).



Certified according to the guidelines for medical devices (already since 1996!) as well as current QMS standards. Certification according to Annex IX - Chapter I, Regulation (EU) 2017/745, and Directive 93/42/EEC; DIN EN ISO 13485:2016 (also Taiwan) and MDSAP für Canada, Brazil, Australia, Japan, USA as well as GOST R for Russia, GOST B for Belarus. The requirements for the biocompatibility of our medical devices are based on ISO 10993-1. The tests required for this standard are performed exclusively in accredited laboratories according to EN ISO/IEC 17025. Registered with the Union Data B for Safety in the Supply Chain (RAKCD) as known consignor since February 2012. All company processes are accompanied by occupational protection management.

IMPRESSION TAKING

A-SILICONE

Detaseal® hydroflow lite	p. 11
Detaseal® hydroflow Xlite	p. 11
Detaseal® hydroflow putty	p. 12
Detaseal® hydroflow soft ² putty	p. 12
Detaseal® hydroflow heavy	p. 13
Detaseal® hydroflow mono	p. 13
Detaseal® antilock	p. 14
Detaseal® function	p. 15
monoprint supra	p. 18
onetime perfect	p. 18
silaplast Futur A	p. 19
silasoft® Futur A	p. 19

BITE REGISTRATION

Detaseal® bite	p. 15
greenbite apple	p. 16
greenbite colour	p. 16
milkbite	p. 17
glassbite	p. 17

C-SILICONE

silaplast Futur	p. 21
Katalysatoren	p. 21
silasoft® direct	p. 22
silasoft® Special	p. 22
silasoft® Normal	p. 23
silone®	p. 23
sta-seal	p. 23
sta-seal f	p. 24
hydro C	p. 25
exaplast	p. 25



CORRECTION / WASH		Viscosity		Flowability		Setting characteristics		Working time min. approx.	Time in the mouth min. approx.	Impression technique	combinable with	Features
		low	high	low	high	slow	fast					
Detaseal® lite regular set		light bodied		high		regular set		2:00	2:00	Correction & double mix technique	Detaseal® putty, Detaseal® heavy	Hydrophilic activity, thixotropic, excellent wetting of humid areas
Detaseal® lite fast set		light bodied		high		very fast set		1:00	1:30	Correction & double mix technique	Detaseal® soft ² putty, Detaseal® putty, Detaseal® heavy	Hydrophilic activity, thixotropic, excellent wetting of humid areas
Detaseal® Xlite regular set		ultra light bodied		very high		regular set		2:00	2:00	Correction & double mix technique	Detaseal® putty, Detaseal® heavy	Hydrophilic activity, very good flow properties, excellent wetting of humid areas
Detaseal® Xlite fast set		ultra light bodied		very high		very fast set		1:00	1:30	Correction & double mix technique	Detaseal® soft ² putty, Detaseal® putty, Detaseal® heavy	Hydrophilic activity, very good flow properties, excellent wetting of humid areas
onetime wash		light bodied		high		fast set		1:30	2:00	Double mix technique	onetime perfect putty, onetime perfect Jumbo, monoprint supra	System for the one step technique, snap set
Detaseal® antilock		light bodied		high		very fast set		1:00	1:20	Double mix & correction technique, single phase impression		Spacer Silicone supersoft, 10 Shore A, thin flowing, highly elastic, easy to demould, tear resistant, fast-curing
silasoft® Futur A		light bodied		high		very fast set		1:00	1:20	Correction & double mix technique	silaplast Futur A	VPS variant of the classic, excellent flow properties

BITE REGISTRATION

Detaseal® bite		medium bodied		medium		regular set		0:45	1:30	Bite key & intraoral fixations		90 Shore A
greenbite apple		medium bodied		medium		fast set		0:30	1:00	Bite key & intraoral fixations		45 Shore D, apple flavoured
greenbite colour		medium bodied		medium		very fast set		0:30	0:50	Bite key & intraoral fixations		45 Shore D, with colour change
milkbite		medium bodied		medium		very fast set		0:30	0:50	Bite key & intraoral fixations		90 Shore A, light creamy consistency, scannable
glassbite		medium bodied		medium		fast set		0:30	1:00	Bite key & intraoral fixations		80 Shore A, transparent, suitable as matrix material during light curing of composites


Detaseal® hydroflow lite



Hydroflow precision impression silicone, easy-flow consistency, ultimate hydrophilicity, highest possible wetting capacity immediately after starting mixing (contact angle < 10°). Exact reproduction of the finest details, particularly in moist regions. „Perfect Flow“ – thixotropic setting with outstanding flow capacity, thermosensitive setting characteristics (SnapSet) for short oral dwelling time, maximum elastic properties after stretching, highly elastic when demoulding, extremely tear resistant. Medical device Cl. IIa, colour: Base pink, Catalyst white 1:1. In 2 setting types:

normal setting – regular set

fast setting – fast set

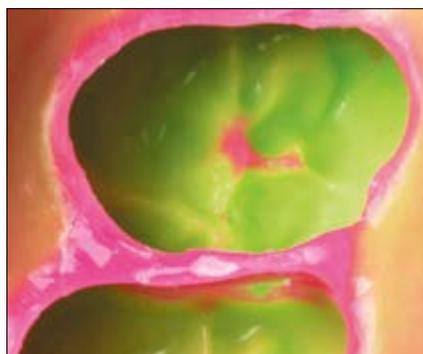

Art.-No.	Packing	Unit
02730	Standard packing regular set base + catalyst mixing cannulas, yellow	2 x 50 ml 12 pcs.
02732	Multi packing regular set base + catalyst mixing cannulas, yellow	4 x 50 ml 24 pcs.
02733	Standard packing fast set base + catalyst mixing cannulas, yellow	2 x 50 ml 12 pcs.
02740	Multi packing fast set base + catalyst mixing cannulas, yellow	4 x 50 ml 24 pcs.
02706	mixing cannulas, yellow	48 pcs.
02712	Intra-Oral-Tips, yellow	96 pcs.


Detaseal® hydroflow Xlite



Hydroflow precision impression silicone, extra light-flowing consistency. Precise detail reproduction, particularly in moist impression regions with ultimate hydrophilicity. Highest possible wetting capacity immediately after starting mixing (contact angle < 10°). „Perfect Flow“ – ideal flow properties, excellent SnapSet, maximum elastic properties – no stretching deformation when demoulding, extremely tear-resistant. Medical device Cl. IIa, colour: Base blue, Catalyst white 1:1. Thermoreactive setting characteristics:

normal setting – regular set

fast setting – fast set

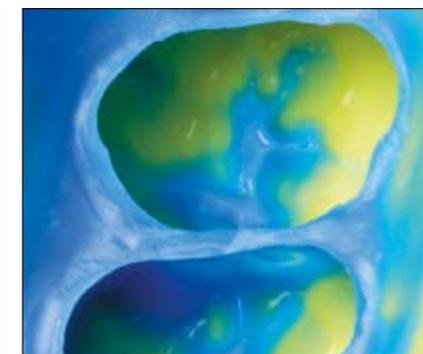

Art.-No.	Packing	Unit
02741	Standard packing regular set base + catalyst mixing cannulas, yellow	2 x 50 ml 12 pcs.
02743	Multi packing regular set base + catalyst mixing cannulas, yellow	4 x 50 ml 24 pcs.
02744	Standard packing fast set base + catalyst mixing cannulas, yellow	2 x 50 ml 12 pcs.
02745	Multi packing fast set base + catalyst mixing cannulas, yellow	4 x 50 ml 24 pcs.
02706	mixing cannulas, yellow	48 pcs.
02712	Intra-Oral-Tips, yellow	96 pcs.

Detaseal® hydroflow putty



Hydroflow impression silicone, kneadable consistency. Smooth initial consistency for perfect mixing results. Ideal for preliminary impressions in the impression technique for corrections and as a tray material in the double-mix technique. Harmonized consistencies of tray and correction materials, no bubbling or flash. Easy to remove due to: high, elastic recovery from deformation with medium final hardness and maximum tensile strength. Time-saving thanks to short mixing time and time in the mouth (only 2 minutes!), neutral tasting, now also for 5:1 mixers; reaches its high, typically kneadable consistency immediately after filling the tray. Medical device Cl. IIa, colour: Base apple green, Catalyst white 1:1/jumbo 5:1



Art.-No.	Packing	Unit
02727	Standard packing putty jar of base + catalyst measuring scoops	2 x 250 ml 2 pcs.
02728	Multi packing putty jar of base + catalyst measuring scoops	4 x 250 ml 2 pcs.
03283	Jumbo packing putty base + catalyst dynamic mixers, yellow rotating ring, yellow	380 ml 10 pcs. 1 pc.
03385	Double Jumbo packing putty base + catalyst dynamic mixers, yellow rotating ring, yellow	2 x 380 ml 2 pcs. 1 pc.


Detaseal® hydroflow soft² putty



Hydroflow silicone impression material, kneadable consistency. Fluffy-soft initial consistency with simultaneously reduced final hardness (60 Shore A). For all precision impression techniques in combination with Detaseal® hydroflow lite/Xlite. Mixing & demoulding with unprecedented ease. Time savings due to a shorter curing time (2:45 min.) with a sufficient processing time of 1:15 min. Maximum patient comfort with an especially short intraoral curing time (1:30 min.!). Easy removal and pleasant mango aroma. Medical device Cl. IIa, colour: Base bright yellow, Catalyst white 1:1



Art.-No.	Packing	Unit
02880	Standard packing soft² putty jar of base jar of catalyst measuring scoops	500 ml 500 ml 2 pcs.

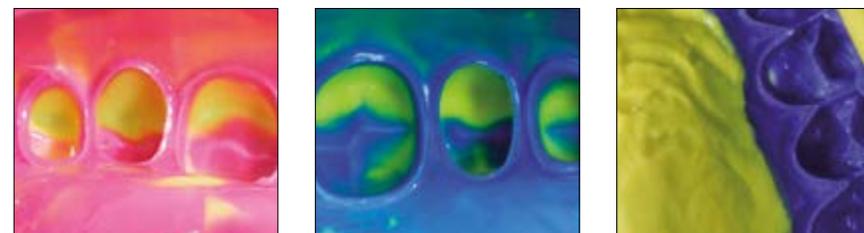

Detaseal® hydroflow heavy



Hydroflow impression silicone, heavy bodied consistency, hydrophilic, stable, easy to remove because of medium final hardness and high elastic recovery from deformation. Fast set, time in mouth 2 minutes. Ideal as a tray material for double-mix / sandwich technique in combination with Detaseal® hydroflow lite/Xlite. Harmonized consistencies, dimensional and volume stability, 5:1 hard cartridges for use with dynamic mixers. Medical device Cl. IIa, colour: Base apple green, Catalyst white 5:1



Art.-No.	Packing	Unit
02729	Jumbo packing heavy base + catalyst dynamic mixers, yellow rotating ring, yellow	380 ml 10 pcs. 1 pc.
03388	Double Jumbo packing heavy base + catalyst dynamic mixers, yellow rotating ring, yellow	2 x 380 ml 2 pcs. 1 pc.
02833	Dynamic mixers, yellow	40 pcs.
02836	Rotating ring, yellow	1 pc.

Detaseal® hydroflow mono



Hydroflow mono-phase impression silicone, medium bodied consistency for single step impression-taking or mixed technique, with distinct snap set. Constant precision, especially in humid areas, due to hydrophilic activation of the surface. Highly thixotropic, flows under pressure. Very tear-resistant with high, elastic recovery from deformation (99.8%), easy to remove. Fast set (total 3:30 min.), dimensional and volume stability, scannable. New: now also in 5:1 hard cartridges for use with dynamic mixers. Medical device Cl. IIa, colour: Base bright yellow, Catalyst white 1:1/jumbo 5:1



Art.-No.	Packing	Unit
02754	Standard packing base + catalyst mixing cannulas, green	2 x 50 ml 6 pcs.
02757	Multi packing base + catalyst mixing cannulas, yellow	4 x 50 ml 12 pcs.
03373	Jumbo packing mono base + catalyst dynamic mixers, yellow rotating ring, yellow	380 ml 10 pcs. 1 pc.
03386	Double Jumbo packing mono base + catalyst dynamic mixers, yellow rotating ring, yellow	2 x 380 ml 2 pcs. 1 pc.


Detaseal® antilock



VPS Spacer-Silicone, thin-flowing, soft consistency. Facilitates reliable demoulding and removal from the mouth, also in case of pronounced retention areas in the impression zone. For versatile use in undercut regions (wide open interdental spaces, pontic bridges or bar constructions, loose teeth with reduced stability, etc.). Short dwell time in the mouth, maximum recovery following expansion, highly elastic during demoulding, tear-resistant. Medical device Cl. IIa, colour: Base pink, catalyst white 1:1



Art.-No.	Packing	Unit
03535	Standard packing base + catalyst mixing cannulas, yellow Intra-Oral-Tips, yellow	50 ml 6 pcs. 6 pcs.

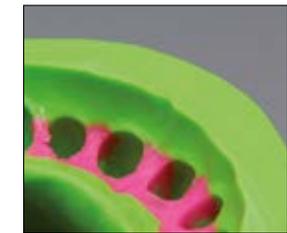

Spacer-Silicone
to block out undercuts

Detaseal® function



Precision impression material for functional impressions and functional border moulding. Addition curing, putty. Ideal smooth and supple consistency, easy to mould, well retaining its shape and volume. With defined plastic phase and prolonged setting time in the mouth. Optimum mixing check due to contrasting colours. Medical device Cl. IIa, colour: Base red, Catalyst yellow 1:1



Art.-No.	Packing	Unit
02521	Standard packing	
	dispenser of base	80 ml
	dispenser of catalyst	80 ml
	moulding syringe	1 pc.
03098	trayloc® A, bottle with brush	17 ml
03392	trayloc® A, vaporizer of	15 ml
02802	moulding syringes/application syringes	10 pcs.
03069	easy application syringe, short	1 pc.

Detaseal® bite



Bite registration material, medium bodied consistency, stays on the impression or preparation. Direct application onto teeth, does not flow into interdental spaces. Easy to reseat due to neon colour, allows easy trimming (cut and fraise), high dimensional stability, fast setting, very high final hardness, approx. 90 Shore A. Medical device Cl. IIa, colour: Base yellowish green, Catalyst white 1:1



Art.-No.	Packing	Unit
02139	Standard packing	
	base + catalyst	2 x 50 ml
	mixing cannulas, rose	8 pcs.
	contouring tips	8 pcs.
02020	Eco packing	
	base + catalyst	8 x 50 ml
	mixing cannulas, rose	32 pcs.
	contouring tips	32 pcs.
02611	contouring tips	12 pcs.

IMPRESSION TAKING A-SILICONES

greenbite apple



Bite registration material, A-silicone based, medium bodied consistency, thixotropic. Direct application onto teeth, does not flow into interdental spaces, easily removed from mouth. Short time in the mouth (60 sec.), maximum final hardness, approx. 45 Shore D, easy processing. Medical device Cl. IIa, colour: Base green, Catalyst yellow 1:1. With green apple flavour!



Art.-No.	Packing	Unit
02641	Standard packing	
	base + catalyst	2 x 50 ml
	mixing cannulas, green	8 pcs.
02647	Eco packing	
	base + catalyst	8 x 50 ml
	mixing cannulas, green	32 pcs.
	contouring tips	32 pcs.
02705	mixing cannulas, green	48 pcs.
02648	contouring tips	12 pcs.

greenbite colour



Bite registration material, A-silicone based, with visual setting check. The colour change from green to yellow indicates that the setting process is completely finished. Medium bodied consistency, thixotropic. Direct application onto teeth. Short time in the mouth (50 sec.). Maximum final hardness, approx. 45 Shore D. Easy processing (cutting, grinding). Medical device Cl. IIa, colour: Base blue, Catalyst yellow 1:1. With apple flavour!



Art.-No.	Packing	Unit
02207	Standard packing	
	base + catalyst	2 x 50 ml
	mixing cannulas, green	8 pcs.
02726	Eco packing	
	base + catalyst	8 x 50 ml
	mixing cannulas, green	32 pcs.
	contouring tips	32 pcs.
02705	mixing cannulas, green	48 pcs.
02648	contouring tips	12 pcs.



milkbite



VPS bite registration, resistance-free bite registration with particularly light-creamy, airy initial consistency. Thixotropic formulation, stable, extra low extrusion pressure. Makes identification and fixation of the natural bite position without interference significantly easier. Time-saving direct application to the dentition, does not flow away into the interdental spaces, easily removed from mouth. Convenient processing time and extra short oral dwelling time (approx. 50 sec.), patient-friendly. Fracture-resistant, can be milled and cut, perfectly scannable. Final hardness: approx. 90 Shore A, medical device Cl. IIa, colour: creamy white 1:1, aroma: ice-cream



Art.-No.	Packing	Unit
03380	Standard packing base + catalyst mixing cannulas, rose contouring tips	2 x 50 ml 8 pcs. 8 pcs.
03381	Eco packing base + catalyst mixing cannulas, rose	8 x 50 ml 32 pcs.
02707	mixing cannulas, rose	48 pcs.
02648	contouring tips	12 pcs.

milkbite
light-creamy consistency
perfectly scannable

glassbite



Bite registration material, A-silicone based. Medium bodied consistency, thixotropic. Direct application onto teeth. Short time in the mouth (60 sec.). High final hardness, approx. 80 Shore A. Also suitable as matrix material during light curing of composites or during positioning of auxiliary parts, x-ray-transparent, easy working on (cutting, grinding). Medical device Cl. IIa, colour: transparent 1:1



Art.-No.	Packing	Unit
02571	Standard packing base + catalyst mixing cannulas, green contouring tips	50 ml 4 pcs. 4 pcs.
02572	Multipack base + catalyst mixing cannulas, green contouring tips	4 x 50 ml 16 pcs. 16 pcs.
02705	mixing cannulas, green	48 pcs.
02648	contouring tips	12 pcs.

IMPRESSION TAKING A-SILICONES

monoprint supra



VPS transfer silicone monophase, medium bodied consistency, with activated surface, specially designed for implantology and fixation impressions with combined prosthetics. Free-flowing, thixotropic setting for easy coverage of abutments. Secure fixation of transfer aids by fast hardening at the end of setting, thermosensitive setting behaviour. Easily removed from mouth, high elasticity. Final hardness approx. 70 Shore A, medical device Cl. IIa, colour: Base neonyellow, Catalyst white 1:1



Art.-No.	Packing	Unit
02525	Standard packing base + catalyst mixing cannulas, green	2 x 50 ml 6 pcs.
02705	mixing cannulas, green	48 pcs.

onetime perfect



Two-phase, one-step A-silicone impression system, for the one-step technique. Extremely short intraoral setting time (2 min.) due to thermally controlled setting. Simultaneous application of putty and heavy material together with wash in only one working stage. Medical device Cl. IIa.

Putty Very high consistency, kneadable, fast setting, 1:1

Heavy High consistency, fast set, 5:1

Wash Low consistency, light bodied. Super hydrophilic, thixotropic, very crisp detail, 1:1
Contrasting colours: Putty and Heavy – turquoise, Wash – violet



Art.-No.	Packing	Unit
03288	Putty – Standard packing jar of base + catalyst measuring scoops	2 x 250 ml 2 pcs.
03289	Wash – Standard packing base + catalyst mixing cannulas, rose Intra-Oral-Tips, white	2 x 50 ml 12 pcs. 12 pcs.
02155	Putty – Eco packing jar of base + catalyst measuring scoops	8 x 250 ml 4 pcs.
02152	Wash – Eco packing base + catalyst mixing cannulas, rose Intra-Oral-Tips, white	8 x 50 ml 48 pcs. 48 pcs.
03102	Jumbo packing heavy base + catalyst dynamic mixers, yellow rotating ring, yellow	380 ml 6 pcs. 1 pc.
02833	Dynamic mixers, yellow	40 pcs.

silaplast Futur A



Kneadable VPS impression material, normal setting, kneadable consistency. With the familiar soft smoothness of silaplast Classic and the ideal final hardness of 63 Shore A. Excellent transfer effect of the initial impression in correction technique or as tray material in double mix technique. Particularly easy and fast homogeneous mixing, high elasticity, easy to demould. Mixing ratio 1:1, contrast colours allow visual mixing control. Medical device Cl. IIa, colours: base white, catalyst blue, spearmint flavour



Art.-No.	Packing	Unit
02335	Standard packing	
	jar of base	450 ml
	jar of catalyst	450 ml
	measuring scoops	2 pcs.

silasoft® Futur A



VPS precision impression material, easy flowing consistency. For correction technique and double mix technique, standard setting. Exact detail reproduction, highly elastic, extremely tear-resistant and with long-term dimensional stability. Thixotropic, perfect flow properties during positioning of impression. Highly elastic during demoulding, safe control, well readable. Easy to dose and mix, mixing ratio 1:1. Medical device Cl. IIa, colours: base pink, catalyst blue



Art.-No.	Packing	Unit
02416	Standard packing	
	base, tube of	90 ml
	catalyst, tube of	90 ml

MATRIX C-SILICONES

TRAY	Image	Viscosity		Flowability		Setting characteristics		Working time min. approx.	Time in the mouth min. approx.	Impression technique	combinable with	Features
		low	high	low	high	slow	fast					
silaplast Futur		very heavy bodied ▾		putty ▾		regular set ▾		1:15	2:30	Correction & double mix technique	silasoft® N, silasoft® S, silasoft® direct	Catalyst paste or liquid
hydro C putty		very heavy bodied ▾		putty ▾		regular set ▾		1:15	3:30	Correction & double mix technique	hydro C wash	Smooth & supple, especially for the mix technique
exaplast putty		very heavy bodied ▾		putty ▾		regular set ▾		1:15	3:30	Correction & double mix technique	exasoft wash	Universally applicable for all impression techniques, soft smooth initial consistency
silone®		medium bodied ▾		medium ▾		regular set ▾		1:15	2:30	Monophase	silasoft® N, silasoft® S, silasoft® direct	Universal, for dentulous impressions, partial conditions
sta-seal		medium bodied ▾		medium ▾		regular set ▾		1:15	2:30	Monophase		For edentulous impressions, flabby ridges, atrophic conditions
CORRECTION / WASH												
silasoft® N		light bodied ▾		high ▾		regular set ▾		1:15	2:30	Correction & double mix technique	silaplast Futur	Excellent flow properties, flows under pressure into the smallest details, with catalyst paste or liquid
silasoft® S		light bodied ▾		high ▾		regular set ▾		1:30	2:30	Correction & double mix technique	silaplast Futur	For manual dosing
silasoft® direct		light bodied ▾		high ▾		regular set ▾		1:15	3:00	Correction & double mix technique	silaplast Futur	Automix2 system 10:1, manual dosing and mixing is omitted
hydro C wash		light bodied ▾		high ▾		regular set ▾		1:15	2:30	Correction & double mix technique	hydro C putty	Hydrophilic, high elasticity
exasoft wash		light bodied ▾		high ▾		regular set ▾		1:15	2:30	Correction & double mix technique	exaplast putty	Excellent „easy flow“ flow properties, hydrophilic
FUNCTIONAL IMPRESSION												
sta-seal f		heavy bodied ▾		stable ▾		slow set ▾		2:30	4:00	Functional impressions for the complete prosthetics, functional border moulding	silasoft® N	With defined plastic phase (2 min. 30 sec.), special catalyst liquid, smooth & supple consistency

silaplast Futur



Putty – kneadable consistency, smooth initial consistency – ideal final hardness and elastic strength. Multi-purpose use, e.g. preliminary impression material for corrections of preparations, overimpression of copperband impressions. New type of formula, less shrinkage than common C-silicones. Medical device Cl. IIa, colour: white



Art.-No.	Packing	Unit
02001	Standard packing jar	900 ml
02002	4-pack jars	4 x 900 ml
02003	Clinic packing bucket	1 x 5.400 ml
02681	Refill bag for jar	900 ml
02015	cat p universal Futur, paste suitable for silaplast, silasoft® Normal, silasoft® Special, silone®, sta-seal	5 x 35 ml
02016	silaplast Futur cat f, liquid suitable for silaplast	5 x 50 ml

Catalysts



Quality catalysts for C-silicones, with new hardener matrix, hypoallergenic, pleasant odour and taste. Blue coloured for exact measuring and mixing control. Available in packs of 5 pieces. Medical device Cl. IIa.



Art.-No.	Packing	Unit
02015	cat p universal Futur, paste suitable for silaplast, silasoft® Normal, silasoft® Special, silone®, sta-seal	5 x 35 ml
02016	silaplast Futur cat f, liquid suitable for silaplast	5 x 50 ml
02017	silasoft® cat f, liquid suitable for silasoft® Normal, silasoft® Special	5 x 10 ml
02018	sta-seal cat f, liquid suitable for silone®, sta-seal, sta-seal f	5 x 10 ml

IMPRESSION TAKING C-SILICONES

silasoft® direct



Precision impression material for correction of preliminary impressions. Light bodied consistency, normal setting. Direct detail reproduction due to its excellent flow properties, hydrophilic. It flows under pressure into the smallest details. Optimal elastic recovery, exceptional tear strength and dimensional stability. Available with the Automix2 cartridge system 10:1, for direct application. Medical device Cl. IIa, colour: Base violet, Catalyst blue



Art.-No.	Packing	Unit
02809	Standard packing base + catalyst 10:1 mixing cannulas, yellow 10:1 Intra-oral-tips, yellow	2 x 50 ml 12 pcs. 12 pcs.
02810	Eco packing base + catalyst 10:1 mixing cannulas, yellow 10:1 Intra-oral-tips, yellow	8 x 50 ml 48 pcs. 48 pcs.
02807	Mixing cannulas 10:1, yellow	48 pcs.
02712	Intra-Oral-Tips, yellow	96 pcs.
02808	Mixing gun for standard cartridges 10:1	1 pz.

silasoft® Special



Precision impression material, with optimised formula, light bodied consistency, for correction of preliminary impressions, for the double impression and double mix technique. Optimum flow properties – the material flows freely when spatulated or syringed. It stays on the impression or preparation. High elasticity, precise detail reproduction and exceptional tear strength. Medical device Cl. IIa, colour: Base pink-transparent, Catalyst blue



Art.-No.	Packing	Unit
02275	Standard packing tube	160 ml
02276	4-pack tubes	4 x 160 ml
02015	cat p universal, paste suitable for silaplast, silasoft® Normal, silasoft® Special, silone®, sta-seal	5 x 35 ml
02017	silasoft® cat f, liquid suitable for silasoft® Normal, silasoft® Special	5 x 10 ml

silasoft® Normal



Precision impression material, light bodied consistency, new type of formula, for correction of preliminary impressions, for the double-impression and double-mix technique. Exact detail reproduction, high elasticity, dimensional stability and tear-strength. Also available in a practical plastic bottle for long-term storage, continuous flow of material and exact dosage. Medical device Cl. IIa, colour: rose-opaque



Art.-No.	Packing	Unit
02004	Standard packing tube	160 ml
02005	4-pack tubes	4 x 160 ml
02320	Standard packing bottle	160 ml
02321	4-pack bottles	4 x 160 ml
02015	cat p universal, paste suitable for silaplast, silasoft® Normal, silasoft® Special, silone®, sta-seal	5 x 35 ml
02017	silasoft® cat f, liquid suitable for silasoft® Normal, silasoft® Special	5 x 10 ml

silone®



Universal precision impression material, medium bodied consistency, for all dentulous impressions. Especially recommended for partial-, copperband- and inlay impressions. Medical device Cl. IIa, colour: red-brown

sta-seal



Precision impression material, medium bodied consistency, for edentulous impressions, especially recommended for atrophic conditions, flabby ridges and relin impressions. Medical device Cl. IIa, colour: lime green



Art.-No.	Packing	Unit
02008	silone® Standard packing tube	160 ml
02009	silone® 3-pack tubes	3 x 160 ml
02010	sta-seal Standard packing tube	160 ml
02011	sta-seal 3-pack tubes	3 x 160 ml
02015	cat p universal, paste suitable for silaplast, silasoft® Normal, silasoft® Special, silone®, sta-seal	5 x 35 ml
02018	sta-seal cat f, liquid suitable for silone®, sta-seal, sta-seal f	5 x 10 ml

IMPRESSION TAKING C-SILICONES

sta-seal f



Special impression material, heavy bodied consistency, for functional impressions and functional border moulding. Excellent smooth initial consistency, easy to mould, ideal final hardness. The prolonged setting time in the mouth facilitates a convenient muscle trimming.

Medical device Cl. IIa, colour: Base green, Catalyst blue

Art.-No.	Packing	Unit
02012	Standard packing measuring syringes sta-seal cat f bottle	3 x 80 ml 1 x 10 ml
02013	Large packing measuring syringes sta-seal cat f bottles	9 x 80 ml 3 x 10 ml
03069	easy application syringe, short	1 pc.
02802	moulding syringes	10 pcs.
02018	sta-seal cat f, liquid suitable for silone®, sta-seal, sta-seal f	5 x 10 ml



hydro C



Precision impression material based on C-silicone for two-step impression technique and two-step putty-wash technique, hydrophilic. Visual check of measuring and mixing with Colour-Safety system. Also suitable for anatomical and reline impressions, bite wafers. Pleasant apple aroma. Medical device Cl. IIa

Putty Kneadable consistency, dimensionally stable. Soft, smooth initial consistency, ideal final hardness.

Wash Light bodied consistency. Tearresistant, highly elastic, very crisp detail, hydrophilic.

Activator Hypoallergenic.

Contrast colours: Putty pastel-green, Wash rose-opaque, Activator blue



Art.-No.	Packing	Unit
02869	Set	
	Putty, jar	1.000 ml
	Wash, tube	150 ml
	Activator, tube	60 ml
02242	Standard packing Putty	1.000 ml
02471	Eco packing Putty	5.400 ml
02243	Standard packing Wash	150 ml
02245	Standard packing Activator	60 ml

exaplast



High-precision premium impression material based on C-silicone, universally applicable for all impression techniques. Excellent readability and mixing check with clear contrast colours, patient-friendly „fresh-mint“ aroma. Medical device Cl. IIa

exaplast putty Kneadable consistency, soft smooth initial consistency, ideal final hardness, highly elastic, very dimensionally stable.

exasoft wash Excellent „easy flow“ flow properties, light bodied consistency, hydrophilic. High elastic strength, extremely tear-resistant, exact detail reproduction.

exagel cat Catalyst gel, hypoallergenic.

Colours: Putty yellow, Wash pink, Catalyst blue



Art.-No.	Packing	Unit
03133	Set	
	Putty, jar	910 ml
	Wash, tube	140 ml
	Cat, tube	60 ml
02907	Standard packing Putty	910 ml
02910	Standard packing Wash	140 ml
03125	Standard packing Cat	60 ml

ACCESSORIES IMPRESSION TAKING

AUTOMIX

mixing gun Automix 1 & 2	p. 27
mixing tips Automix 1 & 2	p. 27
Contouring tips	p. 27
Intra-Oral-Tips	p. 27

ACCESSORIES

Anmischblock, Anmischspatel	S. 28
Deta-Cut	S. 28
Inlay-Löffel	S. 29

TRAY ADHESIVE

sili Adhesive	p. 29
sili Solvent	p. 29
trayloc® A	p. 30
Detaseal® Adhesive rapid	p. 30





02699
Automix2 Mixing gun for standard cartridges 1:1 / 2:1



02808
Automix2 Mixing gun for standard cartridges 10:1



02343
Automix1 Mixing gun for standard cartridges 1:1



Automix2

Art.-No.	Packing	Unit
02699	Mixing gun for standard cartridges 1:1 / 2:1	1 pc.
03617	Mixing gun for standard cartridges „T“	1 pc.
02808	Mixing gun for standard cartridges 10:1	1 pc.
02706	Mixing cannulas, yellow	48 pcs.
03615	T-Mixer, yellow	25 pcs.
02712	Intra-Oral-Tips, yellow	96 pcs.
02707	Mixing cannulas, rose	48 pcs.
03614	T-Mixer, rose	25 pcs.
02345	Intra-Oral-Tips, white	96 pcs.
02611	Contouring tips	12 pcs.
02705	Mixing cannulas, green	48 pcs.
03616	T-Mixer, green	25 pcs.
02648	Contouring tips	12 pcs.
02704	Mixing cannulas, violet	48 pcs.
02711	Intra-Oral-Tips, violet	96 pcs.
02708	Mixing cannulas, blue, long	48 pcs.
02709	Mixing cannulas, blue, short	48 pcs.
03617	Mixing gun for standard cartridges „T“	1 pc.
02808	Mixing gun for standard cartridges 10:1	1 pc.
02807	Mixing cannulas 10:1, yellow	48 pcs.
02183	Mixing cannulas 10:1, blue	25 pcs.
04055	T-Mixer 10:1, blue	25 pcs.
02591	Mixing cannulas, brown 4:1, mini-mix	25 pcs.



Automix1

Art.-No.	Packing	Unit
02343	Mixing gun for standard cartridges 1:1	1 pc.
02344	Mixing cannulas, 6 mm	48 pcs.
02442	Mixing cannulas, 9 mm	48 pcs.
02345	Intra-Oral-Tips, white	96 pcs.
02611	Contouring tips	12 pcs.



ACCESSORIES IMPRESSION TAKING

Mixing pad

Mixing pad vellum paper (P) or plastic coated (K), 30 sheets, with scaling, in two sizes, sheets separable. Medical device Cl. I



Art.-No.	Packing	Unit
02031	Mixing pad (P) large	21,0 x 14,8 cm
02032	Mixing pad (P) small	14,5 x 9,5 cm
02033	Mixing pad (K) large	21,0 x 14,8 cm
02034	Mixing pad (K) small	14,5 x 9,5 cm
02036	Mixing spatula, stainless steel, with wooden handle	1 pc.
02037	Mixing spatula, blue	1 pc.
02830	Mixing spatula, white	1 pc.

Deta-Cut

Sterilizable special instrument, curved, for time saving cutting out of thin and equal flow off grooves in preliminary silicone based impressions.



Art.-No.	Packing	Unit
02323	Deta-Cut	1 pc.

Inlay Trays



Inlay trays for making safer impressions, mechanically improved tray retention. Suitable for partial impressions, preliminary impressions with provisionals, inlays, crown- and bridge work. Disposable item, made of plastic material, perforated. Medical device Cl. I, colour: light blue



Art.-No.	Packing	Unit
02042	Trays, large	12 pcs.
02043	Trays, small	12 pcs.
02044	Trays, mixed each	6 pcs.

sili Adhesive



Adhesive for adhesion of condensation curing impression materials to plastic, metal and individual trays, extremely fast drying (approx. 1 min.). Medical device Cl. I, colour: blue-transparent



Art.-No.	Packing	Unit
02051	sili Adhesive, spray can	250 ml
02053	sili Adhesive Fluid, bottle with brush	25 ml

sili Solvent



Solvent for removal of the adhesive and cleaning of trays after impression taking.

Art.-No.	Packing	Unit
02052	sili Solvent, spray can	250 ml

ACCESSORIES IMPRESSION TAKING

trayloc® A



Adhesive for A-silicones, very high adhesion with thincoated surface conditioning, cementfree. Can be applied more easily and economically than conventional adhesives due to its aqueous-like, thin-flowing consistency. For all plastic, metal and individual trays. With colour indicator for visual application control, fast drying (approx. 3 min.). Also available as a spray, for simple, targeted handling. Medical device Cl. I, colour: blue-transparent



Art.-No.	Packing	Unit
03098	Bottle with brush	17 ml
03392	Vaporizer of	15 ml

Detaseal® Adhesive rapid



For adhesion of addition curing impression materials to plastic, metal and individual trays, fast drying (approx. 1-2 min.). Medical device Cl. I, colour: orange-transparent



Art.-No.	Packing	Unit
02324	Bottle with brush	15 ml

for all trays
plastic, metal and
individual trays



SOFT RELINING

COLD CURING

mollosil® plus
mollosil®

p. 32
p. 33

GLOSS VARNISH

Lustrol

p. 33

ACCESSORIES

mollosil® Remover
good morning Spray

p. 32
p. 34

HEAT CURING

PERMALINER
Molloplast® B

p. 70
p. 70

mollosil® plus



Long-term soft relining material for chairside use, A-silicone based, cold curing, biocompatible. The material is directly applied from the cartridge onto the denture base. Strong bonding to the denture acrylic. Also suitable for application with the indirect technique. Primer with colour indicator for better application check. Booklets for patients and grinding set inclusive. Medical device Cl. IIa, colour: gingiva



Art.-No.	Packing	Unit
02353	Standard packing cartridge Automix2 mixing cannulas, pink mollosil® plus Primer, bottle of mollosil® plus Polish, bottle of grinding sleeve, pointed grinding sleeve holder, pointed pipettes	50 ml 7 pcs. 5 ml 2 x 7 ml 1 pc. 1 pc. 2 pcs.
02354	Refill packing cartridge Automix2	50 ml
02436	Standard packing cartridge Automix1 mixing cannulas, 6 mm mollosil® plus Primer, bottle of mollosil® plus Polish, bottle of grinding sleeve, pointed grinding sleeve holder, pointed pipettes	50 ml 7 pcs. 5 ml 2 x 7 ml 1 pc. 1 pc. 2 pcs.
02439	Refill packing cartridge Automix1	50 ml
02440	mollosil® plus Primer	5 ml
02441	mollosil® plus Polish	2 x 7 ml
02631	mollosil® Remover, dropping bottle	10 ml
02615	grinding sleeve, pointed	10 pcs.
02616	grinding sleeve holder, pointed	1 pc.



Primer with
colour indicator

mollosil®



Long-term relining material for chairside use, A-silicone based, permanently soft, cold curing. Secure bonding to the denture acrylic. For manual mixing, mixing ratio 1:1. Also suitable for immediate dentures. Biocompatible, taste- and odourless. Also suitable for application with the indirect technique. Medical device Cl. IIa, colour: gingiva



Art.-No.	Packing	Unit
03005	Standard packing base, tube of catalyst, tube of adhesive, bottle of Lustrol gloss varnish, bottles of pipettes	50 ml 50 ml 5 ml 2 x 6 ml 2 pcs.
02274	Starterkit base, tube of catalyst, tube of adhesive, bottle of Lustrol gloss varnish, bottles of pipettes	30 ml 30 ml 5 ml 2 x 6 ml 2 pcs.
03007	mollosil® Adhesive	5 ml
03008	Lustrol gloss varnish	2 x 6 ml
02631	mollosil® Remover, dropping bottle	10 ml

Lustrol



Silicone based gloss varnish for sealing and smoothing of permanent soft mollosil® and Molloplast® B silicone based relinings, odour- and tasteless, transparent. Mixing ratio 1:1. Medical device Cl. IIa



Art.-No.	Packing	Unit
03008	Standard packing gloss varnish, bottle of catalyst, bottle of pipettes	6 ml 6 ml 2 pcs.



for permanent use
biocompatible
also suitable for application
with the indirect technique

SOFT RELINING ACCESSORIES

good morning spray

Antibacterial spray solution for A-silicone based relinings, inhibits plaque and delays accumulation of oral microorganisms for 24 hours, without any adverse effects on the natural oral flora. Bacteriostatic effect (inhibits bacteria growth) due to the special combination of active substances, with chlorhexidine and ethereal oils.

Art.-No.	Packing	Unit
02284	Standard packing spray bottle	100 ml
02811	5-pack spray bottle	5 x 100 ml

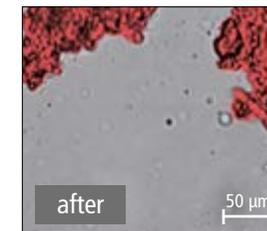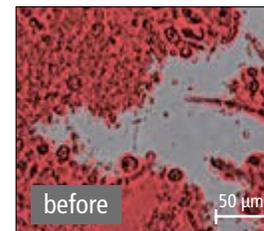



TEMPORARY TREATMENT

C & B COMPOSITE

tempofit® semi 10:1	p. 36
tempofit® premium 10:1	p. 36
tempofit® regular 1:1	p. 37
tempofit® duomix	p. 37
tempofit® finish	p. 39

REPAIR

smartrepair® system	p. 38
smartrepair® rosa	p. 38

FIXATION / IMPLANT CEMENT

tempolink® single	p. 39
tempolink® clear	p. 40
implantlink® semi Classic	p. 40
implantlink® semi Forte	p. 41
implantlink® semi Xray	p. 41
tempofill® 2	p. 42

SEALING

Fermin	p. 42
Detaferm®	p. 42

TEMPORARY TREATMENT C & B COMPOSITE

tempofit® semi 10:1



High-end bis-acrylic-composite for high quality long-term temporary prosthetics, 10:1, self-curing. For fast chairside fabrication of temporary crowns & bridges of highest mechanical strength and stability. Developed for restorations with a wear period > 30 days. With precisely defined elastic phase and minimal polymerization temperature. Esthetic, natural translucence, long-term colour-stable. A particularly thin inhibition layer and smooth surface accelerate finishing. Significant material savings due to use of shortened mixing tips. Neutral taste, direct application with the Automix2 system. Medical device Class IIa, colours: A2, A3, A3.5



Art.-No.	Packing	Unit
04052	Standard packing A2 cartridge (75 g) T-Mixer, blue, 10:1	50 ml 10 pcs.
04053	Standard packing A3 cartridge (75 g) T-Mixer, blue, 10:1	50 ml 10 pcs.
04054	Standard packing A3.5 cartridge (75 g) T-Mixer, blue, 10:1	50 ml 10 pcs.
04055	T-Mixer, blue, 10:1	25 pcs.
02808	Mixing gun for standard cartridges 10:1	1 pc.

Long-term temporary restorations > 30 days
can be shade matched
using composites

tempofit® premium 10:1



High-end bis-acrylic composite 10:1, self curing, for high-quality temporary crowns and bridges and long-term temporaries. Natural dental aesthetics, brilliant translucent shades, even in variable light conditions. Extremely fracture-resistant, highest flexural and abrasion resistance. With a defined elastic phase, safe removal of temporaries, minimal curing temperature. Grinds and polishes extremely well. Neutral tasting and absolutely colour-stable throughout the wearing time. Direct application in the Automix2 system. Medical device Cl. IIa, colours: A1, A2, A3, A3,5, B1, BL



Art.-No.	Packing	Unit
03248	Standard packing A1 cartridge (75 g) mixing cannulas, blue, 10:1	50 ml 10 pcs.
02201	Standard packing A2 cartridge (75 g) mixing cannulas, blue, 10:1	50 ml 10 pcs.
02209	Standard packing A3 cartridge (75 g) mixing cannulas, blue, 10:1	50 ml 10 pcs.
02217	Standard packing A3.5 cartridge (75 g) mixing cannulas, blue, 10:1	50 ml 10 pcs.
04315	Standard packing B1 cartridge (75 g) Mischkanülen, blau 10:1	50 ml 10 St.
04316	Standard packing BL Kartusche (75 g) Mischkanülen, blau 10:1	50 ml 10 St.

tempofit® regular 1:1



Standard bis-acrylic-composite for accurate short and long-term temporary prosthetics, 1:1, self-curing. Natural translucent tooth aesthetics, with long-term colour stabiliser. Highly resistant to bending and breaking, very abrasion-resistant. High initial hardness, short curing time. Easy removal due to elastic phase, low polymerisation temperature. Optimal processing (grinding, polishing), neutral taste. Direct application with the Automix2-System. Medical device Cl. IIa, colours: A1, A2, A3, A3.5



Art.-No.	Packing	Unit
03531	Standard packing A1 cartridge (75 g) mixing cannulas, blue, short	50 ml 10 pcs.
03526	Standard packing A2 cartridge (75 g) mixing cannulas, blue, short	50 ml 10 pcs.
03532	Standard packing A3 cartridge (75 g) mixing cannulas, blue, short	50 ml 10 pcs.
03533	Standard packing A3.5 cartridge (75 g) mixing cannulas, blue, short	50 ml 10 pcs.
02709	mixing cannulas, blue, short	48 pcs.
02699	Mixing gun for standard cartridges 1:1 / 2:1	1 pc.

tempofit® duomix 5:1



Bis-acrylic composite for short- & long-term provisionals 5:1, self curing. Pressure and abrasion resistant, normal setting, highest occlusal fitting accuracy. Handmix in a synchron-dispenser, specially suitable for manual mixing, additional dispenser devices are not required. Exact dosage, without any loss of material in the mixing cannula, not dependent on type of mixing gun. Medical device Cl. IIa, colours: A2, A3



Art.-No.	Packing	Unit
02763	Introkit dispenser 2 x A2, 1 x A3 application syringe 1 mixing spatula, 1 mixing pad	each 25 g each 1 pc.
02106	Single-bag A2 application syringe 1 mixing spatula, 1 small mixing pad	1 x 25 g each 1 pc.
02270	Single-bag A3 application syringe 1 mixing spatula, 1 small mixing pad	1 x 25 g each 1 pc.
02795	Standard packing A2	4 x 25 g
02762	Standard packing A3	4 x 25 g
02800	Eco packing A2	8 x 25 g
02872	Eco packing A3	8 x 25 g
02802	moulding syringes/application syringes	10 pcs.

smartrepair® system



System-set of repair material for tempofit®, light curing, ideal consistency. For fast and precise repairs, delivered in economic direct application syringes with interchangeable application needles. Matched with the colours of tempofit®. With smartbond®, the one-step bonding for absolutely secure adhesion of the repair material. Medical device Cl. IIa, colours: A2, A3, A3.5



Art.-No.	Packing	Unit
02694	smartrepair® set application syringe A2, A3.5 smartbond®, bottle smartbrush application needles flowchart	each 1,5 g 5 ml 5 pcs. 6 pcs. 1 pc.
02772	smartrepair® A2 application needles	1,5 g 5 pcs.
02862	smartrepair® A3 application needles	1,5 g 5 pcs.
02695	smartrepair® A3.5 application needles	1,5 g 5 pcs.
02696	smartbond®, bottle smartbrush	5 ml 5 pcs.
02715	smartbrush	100 pcs.
02714	application needles	10 pcs.

smartrepair® rosa

Modelling and repair resin, light-curing. For aesthetic adjustments and corrections of the gingival region of temporary crowns and bridge restorations or chairside repairs of acrylic prostheses. Universally applicable in practice and laboratory. Saves time with short polymerisation times. Fast, precise application directly from the syringe, economical to use. Ideal modelling properties, thixotropic, MMA-free. Can be polymerised with all standard dental polymerisation lights. Medical device Cl. IIa, colour: pink-transparent



Art.-No.	Packing	Unit
03364	smartrepair® rose application needles	3 g 8 pcs.
02696	smartbond®, bottle smartbrush	5 ml 5 pcs.
02715	smartbrush	100 pcs.
02714	application needles	10 pcs.

smartrepair® System
also for individual shade matching of all
tempofit® C&B composites/FREEPRINT® temp p. 51

tempofit® finish



Light-curing single component varnish for surface sealing of temporary crowns and bridges. "Paint on" polish for chairside use by blue light polymerisation. Brilliant natural finish, without mechanical polishing. Creates a very hard and smooth surface. Also protects against plaque deposits and discolouration. High gloss, colourless, strong bonding. Medical device Cl. IIa



Art.-No.	Packing	Unit
03536	bottle	4 ml
	disposable brush, blue metallic	10 pcs.
03542	disposable brush, blue metallic	100 pcs.

tempolink® single



Temporary fixing cement, polycarboxylate based 1:1, for crowns and bridges, inlays and onlays, zirconium works, self-curing, eugenol-free, thixotropic. Optimum flowability due to its hydrophilic formulation. Secure sealing without any marginal leakage, maximum bonding. No dehydration of the preparation, easy to remove. Just 60 sec. for the fixation of single crowns and smaller applications. Medical device Cl. IIa, colour: natural opaque



Art.-No.	Packing	Unit
02598	Standard packing	
	cartridge mini-mix	5 ml
	mixing cannulas, brown, 1:1	8 pcs.
	flowchart	1 pc.
02761	mixing cannulas, brown, 1:1, mini-mix	25 pcs.

tempolink® clear



Transparent, temporary fixing cement, resin based 4:1, dual curing. Film-forming, therefore highest marginal tightness and easy removal. Distinctly reduced displacement resistance than with common cements. Flexible working times due to optional light curing. Especially suitable for veneers, crowns and bridges in the anterior tooth region, without any aesthetic impairment. Eugenol-free, compatible with all permanent cements. Antibacterial with Triclosan, of indifferent colour and odour. Medical device Cl. IIa, colour: transparent



Art.-No.	Packing	Unit
02593	Standard packing cartridge mini-mix mixing cannulas, brown, 4:1 flowchart	5 ml 8 pcs. 1 pc.
02591	mixing cannulas, brown, 4:1, mini-mix	25 pcs.

implantlink® semi Classic



Cement for fixing long-term temporary dentures on a resin base 4:1, for frictionlocked, chewing-stable cementing of superstructures. Undamaged removal by perfectly matched adhesive properties, significantly lower displacement resistance than conventional cements. Extremely low film thickness (8 µm) for maximum fitting accuracy, excess can be removed easily in large pieces during the plastic phase, does not adhere to soft tissue. Compatible with all material combinations, dual-curing, odour and taste-neutral. Medical device Cl. IIa, colour: natural opaque



Art.-No.	Packing	Unit
03092	Standard packing cartridge mini-mix mixing cannulas, brown, 4:1 flowchart	5 ml 10 pcs. 1 pc.
02591	mixing cannulas, brown, 4:1, mini-mix	25 pcs.



implantlink® semi
Application-Clip!



implantlink® semi Forte



Implant cement for superstructures on a resin base 4:1, specially designed for semipermanent fixing on customised abutments, abutments with reduced adhesion surface or particularly small abutments. With reinforced adhesion and compressive strength. The restoration can still be removed without damage even after extended time in position. Accurate positioning with low displacement resistance and particularly low film thickness. Excess easily removed during the plastic phase. Compatible with all material combinations, dual curing system. Medical device Cl. IIa, colour: natural opaque




Art.-No.	Packing	Unit
03371	Standard packing cartridge mini-mix mixing cannulas, brown, 4:1 flowchart	5 ml 10 pcs. 1 pc.
02591	mixing cannulas, brown, 4:1, mini-mix	25 pcs.



implantlink® semi Xray



Semi-permanent implant cement based on plastic 4:1, for securing suprastructures. Radiopaque with enhanced adhesion and compressive strength, even with extended wear. The restoration can be taken out without damaging it. Exact positioning and highest precision fit due to very thin film thickness. Easy, removal of large pieces of excess in the elastic phase, does not stick to soft tissue. Compatible with all material combinations, dual-hardening, neutral odour and taste. Medical device Cl. IIa, colour: natural opaque

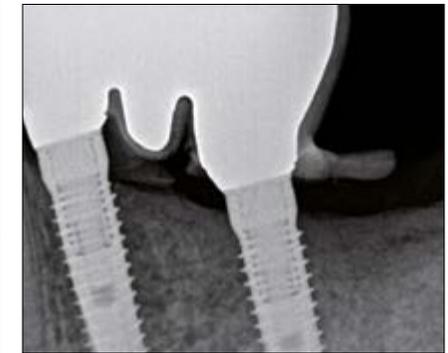

Art.-No.	Packing	Unit
02195	Standard packing cartridge mini-mix mixing cannulas, brown, 4:1 flowchart	5 ml 10 pcs. 1 pc.
02591	mixing cannulas, brown, 4:1, mini-mix	25 pcs.



TEMPORARY TREATMENT SEALING

tempofill® 2



One-part composite material, for temporary treatment of cavities and fillings (inlay/onlay technique and veneers), light curing. Direct, cost-effective dispensing from a syringe. Can be very easily condensed and contoured due to its soft, elastic consistency. Does not stick to the instrument. Cured tempofill® 2 fillings are easily removed without damaging the preparation margins. Medical device Cl. IIa, colour: light-opaque



Art.-No.	Packing	Unit
02117	Standard packing tempofill® 2 inlay syringes	3 x 3 g
02159	Standard packing smartprotect® bottle of smartbrush flowchart	7 ml 20 pcs. 1 pc.
02715	smartbrush	100 pcs.



Fermin



Ready-to-use filling material for provisional sealing. Setting in the mouth under the effect of saliva, great adhesive strength in the cavity, excellent marginal sealing, easy to remove. Medical device Cl. IIa, colour: white opaque

Detaferm®



Ready-to-use filling material for temporary cavity care, with calcium fluoride for the remineralisation of teeth. Setting in the mouth under the effect of saliva, great adhesive strength in the cavity. Simple processing, excellent marginal sealing, easy to remove. Medical device Cl. IIa, colour: white opaque



Art.-No.	Packing	Unit
02054	Fermin, Standard packing jar of 15 ml	40 g
02000	Detaferm®, Standard packing jar of 15 ml	40 g



PREVENTION/ REGENERATION

SEALING OF ROOT DENTINE

smartseal® cervi p. 44

FISSURE SEALING

smartseal® & loc p. 45

smartseal® & loc F p. 45

MOUTH RINSING SOLUTION

smartclean p. 46

PREVENTION/REGENERATION

smartseal® cervi



Sealing varnish to protect exposed, hypersensitive root dentine. Increases mechanical stability, forms an effective erosion barrier and reduces the cervical sensitivity of root dentine. High abrasion resistance due to high degree of linking of highly multifunctional (meth)acrylates. Forms a permanent protective layer against mechanical abrasion and the erosive effects of acids. Structure-reinforcing, desensitising, light-curing. Precise, hygienic application with flexible, fibre-free, plastic micro-applicators. Medical device Cl. IIa



Art.-No.	Packing	Unit
03537	Standard packing dropper bottle applicators, pink disposable dual dishes	5 ml 50 pcs. 24 pcs.
03543	Applicators, pink	100 pcs.



forms an effective erosion barrier
reduces cervical sensitivity



smartseal® & loc / F



Fissure sealant with wet-bonding technology, composite-based for permanent sealing of fissures and enamel defects. Moisture tolerant, hydrophilic. Application on moist or slightly dry enamel surfaces, very effective wetting of tooth enamel. Increased strong bonding due to additional enamel integration of reactive coated fillers. Low viscosity and easy flow for margin-free, tight transitions without cracking or splintering. Absolutely precise direct application with extra long disposable needles. Light-curing, abrasion-proof. Medical device Cl. IIa

smartseal® & loc

Colour: transparent, without fluoride, especially for aesthetic sealing

smartseal® & loc F

Colour: natural opaque, with fluoride



Wet surface welcome!
with or without fluoride

Art.-No.	Packing	Unit
02612	smartseal® & loc Combi Pack smartseal® & loc, syringe smartseal® & loc F, syringe application needles, black	1,5 g 1,5 g 12 pcs.
02573	Standard packing smartseal® & loc application syringes application needles, black	2 x 1,5 g 12 pcs.
02581	Standard packing smartseal® & loc F application syringes application needles, black	2 x 1,5 g 12 pcs.
02456	application needles, black	25 pcs.

PREVENTION DESENSITISATION

smartclean

Mouth rinsing solution for use after professional cleaning and pretreatment, delays formation of plaque. Contains chlorhexidine. Easy, safe use in 5 ml syringes with flexing rinsing cannulas. Non-alcoholic, sugar-free. Refill maxi syringe 50 ml, with integrated adapter for effortless, clean refilling of the 5 ml syringes.

Art.-No.	Packing	Unit
02554	Standard packing syringes rinsing cannulas	4 x 5 ml 4 pcs.
02557	Refill-Container maxi-syringe with integrated adapter	50 ml
02629	rinsing cannulas	25 pcs.



LIQUIDS, SOLUTIONS

LIQUID

Cavidex®

p. 51

SPRAY SOLUTION

good morning spray

p. 51



LIQUIDS, SOLUTIONS

Cavidex®



Topical application for rinsing and drying cavities and dies. Cavidex® facilitates an optimal adhesion of filling materials. Medical device Cl. IIa



Art.-No.	Packing	Unit
02280	Cavidex®, Bottle	150 ml

good morning spray

Antibacterial spray solution for A-silicone based relinings, inhibits plaque and delays accumulation of oral microorganisms for 24 hours, without any adverse effects on the natural oral flora. Bacteriostatic effect (inhibits bacteria growth) due to the special combination of active substances, with chlorhexidine and ethereal oils.

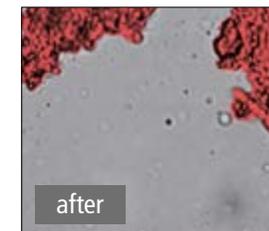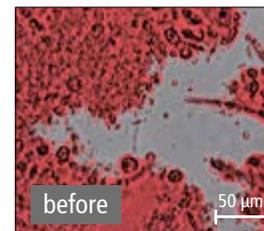

Art.-No.	Packing	Unit
02284	good morning spray, spray bottle	100 ml
02811	good morning spray, 5-pack spray bottle	5 x 100 ml

3D PRINTING

CROWNS & BRIDGES

FREEPRINT® TEMP p. 51

FREEPRINT® CROWN p. 52

DENTURE BASES

FREEPRINT® DENTURE p. 53

FREEPRINT® DENTURE IMPACT p. 54

FREEPRINT® TRYIN p. 54

ORTHODONTIC BASE COMPONENTS

FREEPRINT® ORTHO p. 55

SPLINTS

FREEPRINT® SPLINT 2.0 p. 55

FREEPRINT® SPLINTMASTER p. 56

BRACKET TRANSFER TEMPLATES

FREEPRINT® IBT p. 56

TRAYS & PLATES

FREEPRINT® TRAY 2.0 p. 57

MODEL PRODUCTION

FREEPRINT® MODEL p. 57

FREEPRINT® MODEL 2.0 p. 58

FREEPRINT® MODEL PRO p. 58

FREEPRINT® MODEL T p. 59

FREEPRINT® WW p. 59

GINGIVAL MASKS

FREEPRINT® GINGIVA p. 60

CASTINGS

FREEPRINT® CAST 2.0 p. 60



MATRIX 3D RESINS

Material type	Application	Color	Characteristics	Medical Devices Class MDR	Medical Devices Class FDA	Medical Devices Class NMPA	Material type	Application	Color	Characteristics	Medical Devices Class MDR	Medical Devices Class FDA	Medical Devices Class NMPA
 TEMP	<ul style="list-style-type: none"> Temporary crowns & bridges Temporary anterior and posterior tooth restorations 	A1, A2, A3	<ul style="list-style-type: none"> Natural transparency and tooth esthetics Extremely high construction precision High mechanical stability Biocompatible 	Ila	II	–	 MODEL	<ul style="list-style-type: none"> Dental working and situation models Control models 	Ivory, grey, sand	<ul style="list-style-type: none"> Maximum surface hardness Dimensional stability Comfortable haptic Very good production precision 	TEC resin	TEC resin	I
 CROWN	<ul style="list-style-type: none"> Permanent crowns, denture teeth Long-term temporary bridges 	A1, A2, A3, B1, B3, C2, D3, BL	<ul style="list-style-type: none"> Natural transparency and tooth esthetics Highest abrasion resistance 	Ila	II	–	 MODEL 2.0	<ul style="list-style-type: none"> Dental master and working models Control models 	Caramel, grey, light grey, sand	<ul style="list-style-type: none"> High detail reproduction Maximum surface hardness and dimensional stability Plaster-like appearance and haptic Very good construction precision 	TEC resin	TEC resin	I
 DENTURE	<ul style="list-style-type: none"> Removable denture bases Total prosthesis 	Pink-transparent, pink	<ul style="list-style-type: none"> Long-term stable and biocompatible dentures Fast printing Perfect fit 	Ila	II	–	 MODEL PRO	<ul style="list-style-type: none"> Dental working and situation models Control models 	Caramel, grey, sand	<ul style="list-style-type: none"> Plaster-like haptic Good dimensional stability High flexural strengths Easy post-processing 	TEC resin	TEC resin	in process
 DENTURE IMPACT	<ul style="list-style-type: none"> Removable denture bases Total prosthesis 	Pink-transparent, pink	<ul style="list-style-type: none"> Long-term stable and biocompatible dentures High impact resistance Easy post-processing 	in process	in process	–	 MODEL KFO	<ul style="list-style-type: none"> Laboratory model preparation Orthodontic models 	White	<ul style="list-style-type: none"> Plaster-like haptic Distinctive edge and dimension stability Highest surface quality 	TEC resin	TEC resin	–
 TRYIN	<ul style="list-style-type: none"> Individual functional try-ins 	A2	<ul style="list-style-type: none"> Fast, material-saving production High mechanical stability 	in process	I	–	 MODEL T	<ul style="list-style-type: none"> Working models for thermoforming technique and aligner technology 	Light blue	<ul style="list-style-type: none"> High temperature resistance to process-related temperature stress High edge strength 	TEC resin	TEC resin	I
 ORTHO	<ul style="list-style-type: none"> Surgical guides for implant dentistry Orthodontic base components 	Clear-transparent	<ul style="list-style-type: none"> Very high mechanical stability & construction precision High printing speed Sterilizable Biocompatible 	Ila	I	TEC resin	 MODEL WW	<ul style="list-style-type: none"> Working models for thermoforming technique and aligner technology 	Blue-transparent	<ul style="list-style-type: none"> Water-washable High temperature resistance 	TEC resin	TEC resin	–
 SPLINT 2.0	<ul style="list-style-type: none"> Hard splints 	Clear-transparent	<ul style="list-style-type: none"> High mechanical flexural strength and stability High initial final hardness Biocompatible 	Ila	I	TEC resin	 GINGIVA	<ul style="list-style-type: none"> Flexible gingival masks for dental models 	Gingiva	<ul style="list-style-type: none"> 3D reproduction of functional gingival model segments Excellent elasticity and tear resistance Natural gingiva esthetics 	TEC resin	TEC resin	I
 SPLINTMASTER	<ul style="list-style-type: none"> Flexible splints Retainers Mouthguards Nightguards 	Clear-transparent	<ul style="list-style-type: none"> Flexible High tensile strength High tension-free wearing comfort 	in process	in process	–	 CAST 2.0	<ul style="list-style-type: none"> Dental casting objects for precision casting 	Red-transparent	<ul style="list-style-type: none"> Residue-free burning out High dimensional stability after printing Precise and distortion-free results, even for delicate constructions 	TEC resin	TEC resin	I
 IBT	<ul style="list-style-type: none"> Orthodontic transfer trays for positioning brackets 	Transparent	<ul style="list-style-type: none"> Elastic and tear-resistant Reliable fixing of brackets Biocompatible 	I	I	–							
 TRAY 2.0	<ul style="list-style-type: none"> Individual impression trays Functional impression trays Base resin plates 	Green	<ul style="list-style-type: none"> High dimensional stability, torsional rigidity Max. construction speed Compatible with all impression materials Biocompatible 	I	I	MED resin							


MDR Medical Device Regulation EU FDA Food and Drug Administration USA NMPA National Medical Products Administration China

FREEPRINT® TEMP



Light-curing, biocompatible resin for the 3D printing of temporary crowns & bridges. Exceptional breaking and flexural resistance. The natural-looking translucent shades (according to VITA classical A1–D4 shade guide) can be esthetically modified for single-crown and bridge restorations. High level of oral stability and in conjunction with tempolink®, enable excellent marginal seal during the period of wear. Easy polishing results in very high surface quality with exceptional abrasion resistance. No odour or taste irritations, MMA- & THF-MA-free. Medical device Class IIa, colours: A1, A2, A3

Art.-No.	Packing	Unit
04058	FREEPRINT® TEMP A1	500 g
04059	FREEPRINT® TEMP A2	500 g
04060	FREEPRINT® TEMP A3	500 g
04062	FREEPRINT® TEMP A1	1.000 g
04063	FREEPRINT® TEMP A2	1.000 g
04064	FREEPRINT® TEMP A3	1.000 g

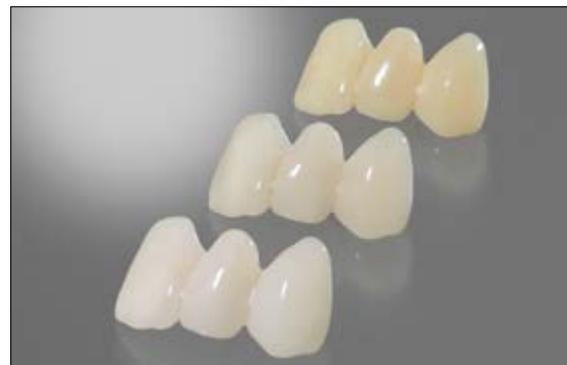

FREEPRINT® TEMP
brilliant, translucent colours!

can be shade matched using the
smartrepair® system and composites p. 38



FREEPRINT®
Application-Clip!



3D PRINTING

FREEPRINT® CROWN




Light-curing formulation for 3D printing of permanent crowns, denture teeth and long-term temporary bridges. Brilliant colors (according to VITA classical A1-D4 shade guide) thanks to perfectly matched transparency and opacity. Optimum dimensional stability of the restorations due to maximum bending and abrasion resistance. No tendency to discolour thanks to low water absorption. Fast, uncomplicated cleaning process. Easy grinding and polishing due to minimal surface chalking. MMA- & THF-MA-free. Medical device Cl. IIa, colour: A1, A2, A3, B1, B3, C2, D3, BL

Art.-No.	Packing	Unit
02372	FREEPRINT® CROWN A1	500 g
02378	FREEPRINT® CROWN A2	500 g
02417	FREEPRINT® CROWN A3	500 g
02481	FREEPRINT® CROWN B1	500 g
02645	FREEPRINT® CROWN B3	500 g
02766	FREEPRINT® CROWN C2	500 g
02783	FREEPRINT® CROWN D3	500 g
02845	FREEPRINT® CROWN BL	500 g
02376	FREEPRINT® CROWN A1	1.000 g
02415	FREEPRINT® CROWN A2	1.000 g
02446	FREEPRINT® CROWN A3	1.000 g
02519	FREEPRINT® CROWN B1	1.000 g
02758	FREEPRINT® CROWN B3	1.000 g
02782	FREEPRINT® CROWN C2	1.000 g
02825	FREEPRINT® CROWN D3	1.000 g
02884	FREEPRINT® CROWN BL	1.000 g



NEW:
FREEPRINT® CROWN
SCIENTIFIC FACTS



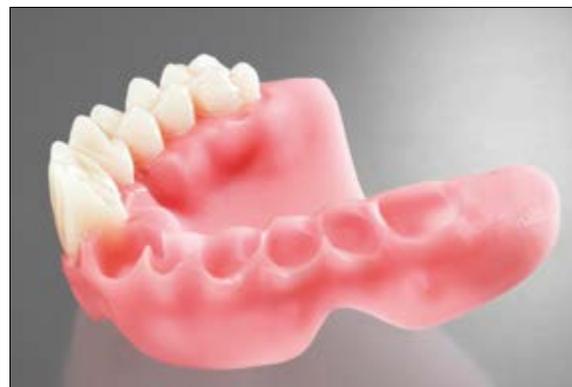

FREEPRINT® DENTURE

Light-curing, biocompatible resin for the 3D printing of removable denture bases. High wearing comfort. Process reliability thanks to high initial hardness following printing. Low viscosity for reduced material consumption and faster cleaning. Very high surface quality, excellent to polish. Displays maximum mechanical flexural and tensile strength without becoming brittle. The rigid denture base withstands high loads in the oral environment. Extremely low shrinkage values compared to PMMA materials. Fully compatible with prefabricated, 3D printed (FREEPRINT® CROWN) or milled teeth. Validated with the VITA VIONIC VIGO System. Natural aesthetics and a light transparency enable alignment with natural gingival color. Odorless and tasteless, MMA- and THF-MA-free, Medical device Cl. IIa, colour: pink-transparent, pink

Art.-No.	Packing	Unit
02060	FREEPRINT® DENTURE pink-transparent	500 g
02040	FREEPRINT® DENTURE pink-transparent	1000 g
04092	FREEPRINT® DENTURE pink	1000 g



NEW:
FREEPRINT®
3D CATALOGUE



FREEPRINT® DENTURE IMPACT

Light-curing formulation for 3D printing of impact resistant denture bases. The increased impact strength provides extremely high fracture stability and ensures lasting functionality over the entire wearing time. Colours and transparency optimally adapted to classic denture resins. cation of fit function and occlusion. Easy to process. Compatible with fabricated, printed (FREEPRINT®CROWN) or milled teeth. MMA-, THF-MA- and TPO-free, tasteless. Medical device Cl. IIa, colour: A2

Art.-No.	Packing	Unit
04436	FREEPRINT® DENTURE IMPACT pink-transparent*	1000 g
04437	FREEPRINT® DENTURE IMPACT pink*	1000 g

*approx. available 2024



FREEPRINT® TRYIN

Light-curing formulation for the 3D printing of individual functional try-ins of digitally manufactured restorations for complete and partial dentures in esthetically pleasing tooth color. Easy verification of fit function and occlusion. Fast and easy additive manufacturing of functional try-ins of individual tooth setups. Easy to process. MMA- & THF-MA-free. Medical device Cl. IIa, colour: A2

Art.-No.	Packing	Unit
04101	FREEPRINT® TRYIN A2*	1.000 g

*approx. available Q3/23



3D PRINTING

FREEPRINT® ORTHO



Light-curing, biocompatible resin for the 3D printing of base parts for orthodontic apparatuses, drilling and X-ray templates, occlusal and fixation splints. Benefits: maximum reliability of construction process, mechanical strength, easy to polish. Breaking strength, elasticity and influence of moisture following the orthodontic standard. Validated processes for autoclave sterilization according to EN ISO 17664! Adaptation of the occlusion field, easy fixation of orthodontic wire elements and auxiliary components with FREEFORM® fixgel. Resistant to oral conditions, MMA-free, neutral odour and taste. Medical device Cl. IIa, colour: clear-transparent

Art.-No.	Packing	Unit
03989	FREEPRINT® ORTHO	1.000 g
03988	FREEPRINT® ORTHO 405	1.000 g



FREEPRINT® SPLINT 2.0



Light-curing, biocompatible resin for the 3D printing of splints, fixation and transfer keys. Hard occlusal splint, rigid type, with high efficiency. Highest mechanical bending and breaking strength, without being brittle. Clear-transparent formulation with high accuracy of fit for visual control in the working area. Process reliability through high initial hardness and final strength. Low-viscosity approach for low material consumption and more rapid cleaning. Compatible with FREEFORM® plast for additional occlusal design in practice. MMA- and THF-MA-free, neutral odour and taste. Medical device Cl. IIa, colour: clear-transparent

Art.-No.	Packing	Unit
02080	FREEPRINT® SPLINT 2.0	500 g
02076	FREEPRINT® SPLINT 2.0	1.000 g





FREEPRINT® SPLINTMASTER

Light-curing formulation for 3D printing of flexible splints, retainers, mouthguards and nightguards. In two levels of flexibility: Taff & Flex

Clear-transparent, flexible and fracture-resistant splints for high, tension-free wearing comfort. Extremely easy to maintain, clean and polish. Wide range of applications, e.g. bite splints, mouth guards, bite plates, snoring devices. MMA-, THF-MA- and TPO-free, tasteless. Medical device Cl. I, colour: transparent

Art.-No.	Packing	Unit
04433	FREEPRINT® SPLINTMASTER TAFF*	1.000 g
04432	FREEPRINT® SPLINTMASTER FLEX*	1.000 g


*approx. available Q4/23



FREEPRINT® IBT

Light-curing, biocompatible resin for the 3D printing of orthodontic transfer trays for positioning brackets. Easy, precise positioning and application of the brackets due to the indirect bonding technique. The transparent bracket transfer templates allow easy visual control. High tensile strength and flexibility allow the easy positioning and subsequent removal of the templates in one single work step. Bisphenol A-, MMA- and THF-MA-free, odorless and tasteless. Medical device Cl. I, colour: transparent

Art.-No.	Packing	Unit
04248	FREEPRINT® IBT	500 g
04249	FREEPRINT® IBT	1.000 g

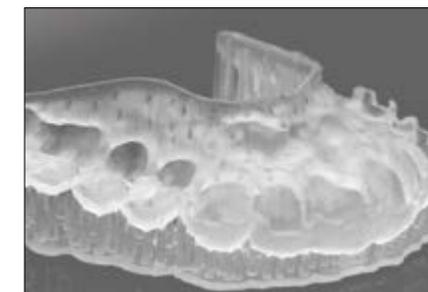

3D PRINTING

FREEPRINT® TRAY 2.0



Light-curing, biocompatible resin for the 3D printing of individual impression and functional trays and base plates. Low viscosity for low material consumption and more rapid cleaning. Highest dimensional stability and torsional rigidity for accurate and distortion-free impression taking. Perfect for implant impression taking within the digital workflow. Printable with 200 µm layer thickness. Compatible for all tray adhesives and impression materials. MMA- and THF-MA-free. Medical device Cl. I, colour: green

Art.-No.	Packing	Unit
02505	FREEPRINT® TRAY 2.0	1.000 g



FREEPRINT® MODEL

Light-curing formulation for 3D printing of dental master and working models. Perfect detail reproduction due to plaster-like colours: grey, ivory, sand. Fast-printing, maximum surface hardness and shape stability. Maximum precision in construction, feel and stability meet the high demands of model production. Opaque colour adjustment for the optical detection of surface structure, preparation margins, etc. Bisphenol A- & MMA-free. Colours: ivory, grey, sand

Art.-No.	Packing	Unit
03778	FREEPRINT® MODEL sand	1.000 g
03780	FREEPRINT® MODEL ivory	1.000 g
03782	FREEPRINT® MODEL grey	1.000 g
03065	FREEPRINT® MODEL 405 sand	1.000 g
03779	FREEPRINT® MODEL 405 ivory	1.000 g
03781	FREEPRINT® MODEL 405 grey	1.000 g



FREEPRINT® MODEL 2.0

Light-curing formulation for 3D printing of dental models, master models, situation and orthodontic models. High detail reproduction, maximum surface hardness and shape stability provide an excellent mechanical strain. Faster workflow due to shortened post-processing. Maximum precision in construction, feel and stability meet the high demands of model production. The distinct edge stability and abrasion resistance make the models comparable to conventional plaster models in terms of handling. Opaque colour adjustment for the optical detection of surface structure, preparation margins, etc. MMA- & THF-MA-free. Colours: caramel, grey, light grey, sand, white


Art.-No.	Packing	Unit
02128	FREEPRINT® MODEL 2.0 sand	1.000 g
02099	FREEPRINT® MODEL 2.0 ight-grey	1.000 g
02177	FREEPRINT® MODEL 2.0 grey	1.000 g
02850	FREEPRINT® MODEL 2.0 caramel	1.000 g
02148	FREEPRINT® MODEL 2.0 white	1.000 g



FREEPRINT® MODEL PRO

Light-curing formulation for 3D printing of dental master and working models. Fast-printing. Precise reproduction of details, maximum surface hardness and shape stability. Maximum precision in construction, feel and stability meet the high demands of model production. The further increase in mechanical strength contributes to optimum functionality. The formulation has been revised according to the latest REACH Regulation. Colours: caramel, grey, sand

Art.-Nr.	Packung	Einheit
04440	FREEPRINT® MODEL PRO caramel	1.000 g
04438	FREEPRINT® MODEL PRO grey	1.000 g
04439	FREEPRINT® MODEL PRO sand	1.000 g

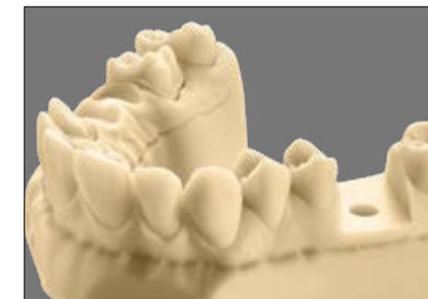

3D PRINTING

FREEPRINT® MODEL T

Light-curing, temperature-resistant resin for the 3D printing of dental models for the thermoforming technique. Precise reproduction of details, maximum surface hardness and edge stability of the models. Precision in construction, tactility and stability meet the high demands of model fabrication. The pronounced intrinsic stability also allows the fabrication of hollow thermoformed models. Low viscosity for considerably easier cleaning and low material consumption. High temperature resistance. The stability of the models is preserved even during heating in thermoforming. MMA-free. Colour: light blue


Art.-No.	Packing	Unit
02332	FREEPRINT® MODEL T	1.000 g
02322	FREEPRINT® MODEL T 405	1.000 g



FREEPRINT® MODEL WW

Light-curing formulation for the 3D printing of dental models for the thermoforming technique. The water-washable material meets all requirements for digitally produced models in thermoforming. No use of solvent necessary. The high edge strength and good intrinsic stability of the material allow production of hollow thermoformed models. High temperature resistance. The stability of the models is not affected by the heat. Cost-efficient model production. MMA- & THF-MA-free. Colour: blue-transparent

Art.-No.	Packing	Unit
03105	FREEPRINT® MODEL WW	1.000 g

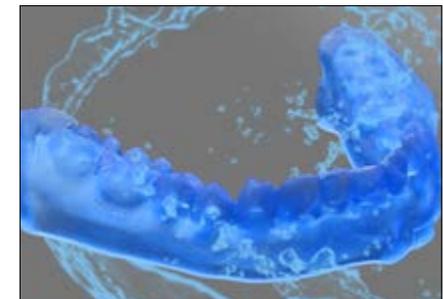

FREEPRINT® GINGIVA

Light-curing for the 3D printing of flexible gingival masks for dental models. For 3D reproduction of functional gingival model segments in a digital workflow in conjunction with FREEPRINT® MODEL. Excellent elasticity and tear-resistance. Excellent dimensional stability, no shrinkage or aging, ductile even when stored over a long period. Natural gingiva esthetics, no annoying or unpleasant odors from the completed gingival masks. Bisphenol A-, MMA- and THF-MA-free. Colour: gingiva

Art.-No.	Packing	Unit
02820	FREEPRINT® GINGIVA	500 g
02843	FREEPRINT® GINGIVA	1.000 g



FREEPRINT® CAST 2.0

Light-curing resin for the 3D printing of high-precision casting objects. Excellent dimensional stability after printing ensures safe application with fit check, in particular where cast models and molds are concerned. Printing results are precise and without any distortions, even for extremely delicate and intricate models. Any corrections or repairs after printing are possible with easyform gel LC. Low viscosity and optimized depth of cure help reduce printing time and material consumption and facilitate faster cleaning. Burns without leaving any residues in the mold and is the basis for high-grade modeling results that ensure the perfect fit. Phosphate-bonded investment materials are used as investments. MMA & THF-MA free. Colour: red

Art.-No.	Packing	Unit
02548	FREEPRINT® CAST 2.0	500 g
02632	FREEPRINT® CAST 2.0	1.000 g



LABORATORY

LIGHT CURING POLYMERS

FREEFORM® p. 62

easyform gel / paste p. 63

INSULATION

FLEXISTONE® p. 64

FLEXISTONE® Plus p. 64

PUTTY

blue eco p. 65

blue eco stone p. 65

blue eco scan® p. 66

compact lab putty p. 66

GINGIVAL MASK

Esthetic Mask p. 67

Implant Mask / scan p. 68

GINGIVAMOLL® p. 71

RELINING

PERMALINER p. 69

Molloplast® B p. 69

Primo p. 70

Tools p. 70

Molloplast® Remover p. 70

RESOURCES

Itopol / Oropol p. 71

xynon p. 72

glad Spray p. 72

modell Spray p. 72





FREEFORM®



Ready-to-use modelling resin, light-curing for multiple indications (e.g., drilling and X-ray templates, splints, base plates, transfer keys). Radiotransparent, can be sterilised and disinfected, biocompatible. Medical device CI. IIa, colour: clear-transparent

plast kneadable paste, can be shaped and modelled, dimension-stable, MMA-free.

fixgel stable gel for direct application from the dosing syringe, MMA-free. High mechanical stability for rapid fixation and integration of drill guides or X-ray reference spheres. Also ideal for reinforcing, individualising and repairing.

bond light-curing bonding agent for PMMA-compatible thermoforming films and acrylates. Also for repairs & revisions of FREEFORM® plast.

coat light-curing, single-component finishing gloss for the final surface sealing. Creates a smooth, hard surface. Brilliant finish, totally without polishing.



Art.-N.	Packing	Unit
02204	Starterset plast, jar fixgel, syringe bond, bottle coat, bottle disposable brush, blue metallic application needles	60 g 3 g 5 ml 10 ml 10 pcs. 10 pcs.
02542	Standard packing plast , jar	60 g
02829	Standard packing fixgel , syringe	3 g
02702	Standard packing bond , bottle	5 ml
02549	Standard packing coat , bottle	10 ml
03542	Disposable brush, blue metallic	100 pcs.
02714	Application needles	10 pcs.



FREEFORM®
Application-Clip



easyform gel / paste

Ready-for-use modelling resin, light-curing, as gel or paste, MMA-free. Very high fitting accuracy for multiple indications, particularly for casting. Burns off without residue, even with thicker layer. Matching viscosities of gel and paste for fast, accurate work. Dimensionally stable, no polymerisation delay, high final hardness and stability. Can be combined with conventional or light-curing wax. Visual layer thickness control with slight translucency. Colour: red

Gel Direct application from the syringe, stable, easily guided for modelling.

Paste Dough-like consistency, easily shaped and modelled.



Art.-No.	Packing	Unit
03376	Standard packing gel syringes application needles	3 x 3 g 10 pcs.
02714	application needles	10 pcs.
03375	Standard packing paste jar of	30 g



Burns off without residue, even
with thicker layer!



FLEXISTONE®

Insulating and modelling material, C-silicone based, light bodied, excellent stability, very high final hardness (75 Shore A). For insulating acrylic or porcelain teeth during investing. Can be easily applied in one single procedure onto acrylic teeth and wax, forming a thin, very hard insulating layer for fixing of teeth during pressing. No retention crystals are required. Colour: green



Art.-No.	Packing	Unit
03009	Standard packing bottle with stand of (350 g) catalyst, bottle of mixing bowl	190 ml 30 ml 1 pc.
03010	Large packing bottles with stand of (5 x 350 g) catalyst, bottle of mixing bowls	5 x 190 ml 3 x 30 ml 2 pcs.
03165	Catalyst, bottle of	30 ml

FLEXISTONE® Plus

Insulating and modelling material, A-silicone based, light bodied, excellent stability, very high final hardness (85 Shore A). Optimum flow properties during spatulation. For insulating dentures during investing. Protection of acrylics and acrylic teeth against plaster. Easy application - 1:1 mixing system. Especially indicated for use with injection moulding techniques, recommended by IVOCLAR for the SR-Ivocap-System. Colour: Base blue, Catalyst white 1:1



Art.-No.	Packing	Unit
02383	Standard packing base + catalyst bottles with stands mixing bowl	2 x 160 ml 1 pc.
02329	Eco packing base + catalyst plastic bottles mixing bowls	2 x 1.000 ml 4 pcs.

LABORATORY PUTTY

blue eco

Putty for the dental laboratory, addition curing, very high, easily kneadable consistency. Precise detail rendition, stable in shape and volume, smooth, heat-resistant up to 200 °C, standard final hardness 86 Shore A. Excellent mechanical properties, easy to cut. For versatile use (secure fixation of artificial teeth, precasts, checkbites, models, etc.). High economy, optimum mixing control by contrast colours. Colour: Base blue, Catalyst beige 1:1



Art.-No.	Packing	Unit
02038	Standard packing	
	base, jar of	800 g
	catalyst, jar of	800 g
	measuring scoops	2 pcs.
02467	Mega pack	
	base, bucket of	5 kg
	catalyst, bucket of	5 kg
	measuring scoops	2 pcs.

blue eco stone

Laboratory putty, A-silicone based, high viscosity, smooth initial consistency. High definition, dimensionally stable, especially smooth, heat-resistance up to 200 °C. Extra high final hardness, approx. 96 Shore A, for versatile use (fabrication of precasts, checkbites, models for denture repairs, blockings-out, etc.), very economical, easy to cut. Colour: Base dark blue, Catalyst white 1:1



Art.-No.	Packing	Unit
03117	Standard packing	
	base, jar of	800 g
	catalyst, jar of	800 g
	measuring scoops	2 pcs.
02517	Eco pack	
	base, jar of	1.400 g
	catalyst, jar of	1.400 g
	measuring scoops	2 pcs.
02985	Mega pack	
	base, bucket of	5 kg
	catalyst, bucket of	5 kg
	measuring scoops	2 pcs.

blue eco scan®

Scannable laboratory putty, addition curing, high, smooth consistency, non sticky. For powderless digital 3D data acquisition of the counter bite. Can be read by all CAD/CAM/CIM systems, optimum exposure quality and artifact-free image reproduction at very high contrast. For all conventional indications of laboratory putty. Low displacement resistance, precise detail rendition, stable in shape and volume, with firm edges, easy to cut. No breaking out of thin contact point areas. Final hardness: 85 Shore A. Colour: Base light grey, Catalyst white 1:1



Art.-No.	Packing	Unit
03118	Standard packing base, jar of catalyst, jar of measuring scoops	800 g 800 g 2 pcs.

compact lab putty

Putty for laboratories, C-silicone based, smooth initial consistency, high final hardness (90 Shore A), thermoresistant. Multi-purpose use (for fixation of teeth, insulation, key to occlusal recording, blocking out, etc.). Variable working and setting time due to individual dosage of catalyst. Exact detail reproduction, high dimensional stability. Colour: mouse grey





Art.-No.	Packing	Unit
02410	Standard packing plastic bucket of (8 kg) measuring scoops	5.400 ml 2 pcs.
02296	cat compact, liquid, bottle of	50 ml
02465	Eco-Set plastic bucket of (8 kg) cat compact, liquid, bottles of measuring scoops	5.400 ml 3 x 50 ml 2 pcs.
02797	Eco-Set plastic bucket of (8 kg) cat compact, paste, tubes of measuring scoops	5.400 ml 5 x 25 ml 2 pcs.
02823	cat compact, paste, tube of	25 ml

Esthetic Mask

Flexible gingival mask, A-silicone based, for the fabrication of master models and working casts in the dental technique, cold curing, high dimensional stability and tear strength. Fast and easy processing, approx. 45 Shore A, mixing ratio of 1:1, with re-usable syringes. For perfect natural aesthetics. Available in tubes and Automix2-cartridges. Colour: gingiva



Art.-No.	Packing	Unit
02340	Standard packing	
	base, tube of	50 ml
	catalyst, tube of	50 ml
	separating liquid for silicones, vaporizer of disposable syringes	15 ml
		2 pcs.
03113	Cartridge packing	
	cartridges	2 x 50 ml
	mixing cannulas, yellow	10 pcs.
	separating liquid for silicones, vaporizer of	15 ml
02706	mixing cannulas, yellow	48 pcs.
02712	Intra-Oral-Tips, yellow	96 pcs.
02690	Separating Agent	
	separating liquid for silicones, vaporizer of	15 ml

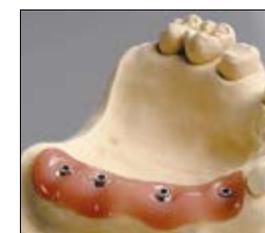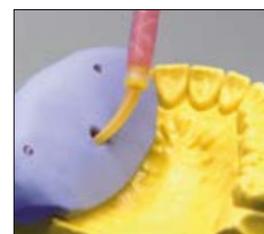

Implant Mask / scan



Addition curing special silicone for manufacturing flexible gingival masks for implant procedures. Free-flowing, dimensionally stable, tear-resistant. Flows evenly around impression coping with free-flowing properties. Very high elasticity allows repositioning as required. Very easily milled with high final hardness, approx. 70 Shore A.

Scan Version, for unpowdered digital 3D recording. Readable by all standard digital scanners, high recording quality for interference-free image reproduction and data processing. Colour: gingiva



Art.-No.	Packing	Unit
02529	Standard packing	
	cartridges mini-mix 1:1	2 x 10 ml
	mixing cannulas, light green, 1:1	6 pcs.
	separating liquid for silicones, vaporizer of	15 ml
03374	Standard packing scan	
	cartridges mini-mix 1:1	2 x 10 ml
	mixing cannulas, light green, 1:1	6 pcs.
	separating liquid for silicones, vaporizer of	15 ml
02605	mixing cannulas, light green 1:1, mini-mix	25 pcs.
03552	Cartridge packing scan	
	cartridges	2 x 50 ml
	mixing cannulas, blue, short	10 pcs.
	separating liquid for silicones, vaporizer of	15 ml
02709	mixing cannulas, blue, short	48 pcs.
02690	Separating Agent	
	separating liquid for silicones, vaporizer of	15 ml

PERMALINER



Ready-to-use hot cure silicone for the permanent lining of dentures. Permanently soft with reduced polymerisation time. Particularly suitable for the flask technique in the dental laboratory, ideal for the manufacturing of new prostheses. Expanded indication spectrum, even with the most difficult jaw conditions. Holds absolutely securely to all acrylates, without adhesive. Reduces the deposit of oral-specific bacteria through an anti-plaque effect. No softening, no embrittlement, remains elastic even after long periods of wear. Biocompatible, MMA-free, neutral odour and taste. Medical device Cl. IIa, colour: light pink



Art.-No.	Packing	Unit
02092	Standard packing jar of	30 g
02146	Large packing jar of	70 g
03153	Pre-polishers (10 pcs. + 1 mandres)	set
03120	Cutters, 2 pcs.	set
03101	Cutters, 3 pcs.	set

50 % shorter
polymerisation time



Molloplast® B
Application-Clip
YouTube

Molloplast® B



Permanent soft relining material for processing in dental laboratories. One-component silicone, heat curing, microwave-polymerising, wide indications (e.g. for obturators, mouth guards, etc.). Improved bonding to new and old denture base acrylics. Optimised consistency due to new vacuum filling technique. Resisting the influences of the oral environment, with anti-plaque effect. It's outstanding properties are clinically tested and documented. Medical device Cl. IIa



Art.-No.	Packing	Unit
03000	Standard packing jar of	45 g
03001	Large packing jar of	170 g
03002	Combi packing jar of Primo Adhesive, bottle of	45 g 15 ml
03004	Primo Adhesive bottle with brush of	15 ml



Primo



Particularly for acrylic dentures, increases and ensures the bonding of Molloplast® B to the denture base. Also especially suitable in case of repeated repairs, extensions and relinings of acrylates. Application by means of the integrated brush. Medical device Cl. IIa



Art.-No.	Packing	Unit
03004	Primo Adhesive bottle with brush of	15 ml

Molloplast® Remover

Dropping solution for fast and secure dissolution of the bonding between elastic relining materials and the denture acrylic. Simple application, without any timeconsuming removing by grinding. No residues, no interaction during repeated relinings.

Bearbeitungswerkzeuge

Grinding sleeves
For smoothing and removing of uneven spots and for final finishing of permanent soft relining materials and silicones.

Cutters & pre-polishers
For optimal trimming of permanent soft, silicone based relinings. No smear effect during cutting, negligent heat development thus no separation of acrylic and relining material. Final finishing with pre-polishers (smoothing and removal of uneven spots).

Art.-No.	Packing	Unit
02206	Molloplast® Remover, dropping bottle	10 ml
03101	Cutters, 3 pcs.	set
03120	Cutters, 2 pcs.	set
03153	Pre-polishers (10 pcs. + 1 mandres)	set
02616	Grinding sleeve holder, pointed	1 pc.
02615	Grinding sleeves, pointed	10 pcs.



LABORATORY

GINGIVAMOLL®

Silicone based epithetic material, heat-curing, for the fabrication of flexible gingival epithesis as a temporary replacement for the receded gingiva – after periodontal treatment. For the correction of aesthetic and phonetic insufficiencies and after implantations, clinically tested. Also available as Starterkit. Medical device Cl. IIa




Art.-No.	Packing	Unit
03053	Starterkit	
	base material, pink/light pink	each 2,5 g
	base material, opaque	4 g
	protective lacquer, bottle of	5 ml
	paint liquid, bottle of	5 ml
	paint brush	1 pc.
	paint colours, blue/red	each 1 pc.

Itopol / Oropol

High gloss polishing pastes, non-greasy, water-soluble and non-smearing.

ITOPOL for cobalt-chromium alloys

OROPOL for precious metals, non-precious metals and acrylic resins

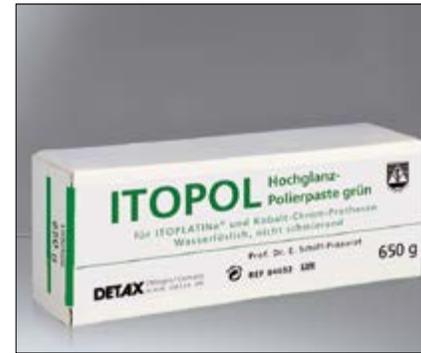

Art.-No.	Packing	Unit
04032	ITOPOL, green, bar	650 g
04033	OROPOL, white, bar	650 g

xynon

Liquid insulating agent. Insulates plaster, resin, metal and porcelain from wax. For application with brushes or for directed spray coating with a cannula. Does not form an oily film on the plaster surface.

glad spray


Wetting agent for the casting technique, for smooth and bubble-free castings. Easy handling by spraying, thus no damage to the cast object.



Art.-No.	Packing	Unit
02077	xynon Spray can with cannula	250 ml
02078	xynon Plastic bottle	100 ml
02079	xynon Plastic bottle	1.000 ml
02074	glad spray Spray can	250 ml

modell spray

Special lacquer for the model cast technique. For smoothing, hardening and sealing of model surfaces. Easy application by spraying instead of dipping.

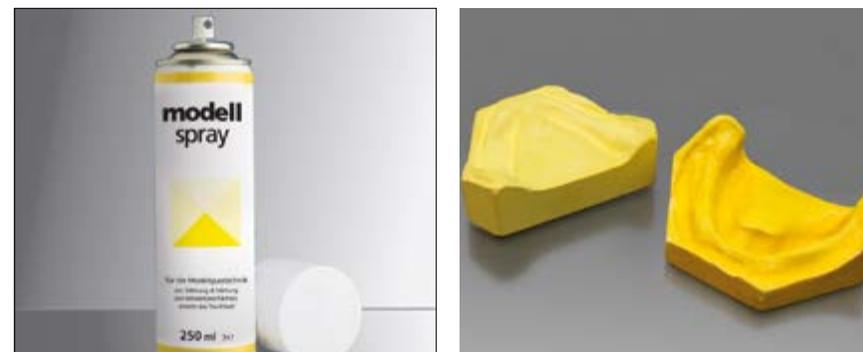

Art.-No.	Packing	Unit
02075	modell spray Spray can	250 ml

INDEX

B

blue eco	65
blue eco scan®	66
blue eco stone	65

C

catalysts	21
Cavidex®	48
compact lab putty	66

D

Deta-Cut	28
Detaferm®	42
Detaseal® Adhesive rapid	30
Detaseal® antilock	14
Detaseal® bite	15
Detaseal® function	15
Detaseal® hydroflow heavy	13
Detaseal® hydroflow lite	11
Detaseal® hydroflow mono	13
Detaseal® hydroflow putty	12
Detaseal® hydroflow soft ² putty	12
Detaseal® hydroflow Xlite	11

E

easyform gel / paste	63
Esthetic Mask	67
exaplast	25

F

Fermin	42
FLEXISTONE®	64
FLEXISTONE® Plus	64
FREEFORM®	62
FREEPRINT® CAST 2.0	60
FREEPRINT® DENTURE	53
FREEPRINT® DENTURE IMPACT	54
FREEPRINT® GINGIVA	60
FREEPRINT® IBT	56
FREEPRINT® MODEL	57
FREEPRINT® MODEL 2.0	58
FREEPRINT® MODEL PRO	58
FREEPRINT® MODEL T	59

FREEPRINT® ORTHO	55
FREEPRINT® SPLINT 2.0	55
FREEPRINT® SPLINTMASTER	56
FREEPRINT® TEMP	51
FREEPRINT® TRAY 2.0	57
FREEPRINT® TRYIN	54

G

GINGIVAMOLL®	71
glad Spray	72
glassbite	17
good morning spray	34
greenbite apple	16
greenbite colour	16

H

hydro C	25
---------------	----

I

Implant Mask / scan	68
implantlink® semi Classic	40
implantlink® semi Forte	41
implantlink® semi Xray	41
Inlay Trays	29
Intra-Oral-Tips	27
Itopol / Oropol	71

L

Lustrol	33
---------------	----

M

milkbite	17
mixing gun Automix 1 & 2	27
mixing pad	28
mixing tips Automix 1 & 2	27
modell Spray	72
Molloplast® B	69
Molloplast® Remover	70
Molloplast® Rotating Tools	70
mollosil®	33
mollosil® plus	32
mollosil® Remover	32
monoprint supra	18

O

onetime perfect	18
-----------------------	----

P

PERMALINER	69
Primo	70

S

silaplast Futur	21
silaplast Futur A	19
silasoft® Futur A	19
silasoft® direct	22
silasoft® Normal	23
silasoft® Special	22
sili Haftlack	29
sili Haftlöser	29
silone®	23
smartclean	46
smartrepair® rosa	38
smartrepair® system	38
smartseal® & loc	45
smartseal® & loc F	45
smartseal® cervi	44
sta-seal	23
sta-seal f	24

T

tempofill® 2	42
tempofit® duomix 5:1	37
tempofit® finish	39
tempofit® premium 10:1	36
tempofit® regular 1:1	37
tempofit® semi 10:1	36
tempolink® clear	40
tempolink® single	39
trayloc® A	30

X

xynon	72
-------------	----

HIGH IMPRESSION TAKING
SOFT RELINING
PREVENTION & REGENERATION

PERFORMANCE PRODUCT VALUE
INNOVATIVE MATERIALS
UNLIMITED POSSIBILITIES

POLYMERS LABORATORY
TEMPORARY TREATMENT
LIQUIDS & SOLUTIONS

DETAX GmbH · Carl-Zeiss-Straße 4 · 76275 Ettlingen/Germany
Tel. +49 7243 / 510-0 · Fax +49 7243 / 510-100 · post@detax.com · www.detax.com

3D RESINS